

Assembly Instructions


StyleView™: SV22 Replacement Battery




WARNING: Only Ergotron-specified batteries may be used in the StyleView Power System. Please call customer care for details.




Do not replace the batteries while cart is located in an oxygen rich or hazardous environment. Sparking may occur.

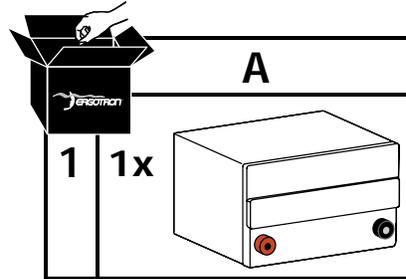


1 a Turn off all mounted equipment.

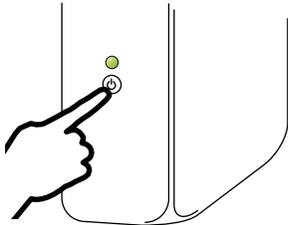


b Disconnect Power System from power source.

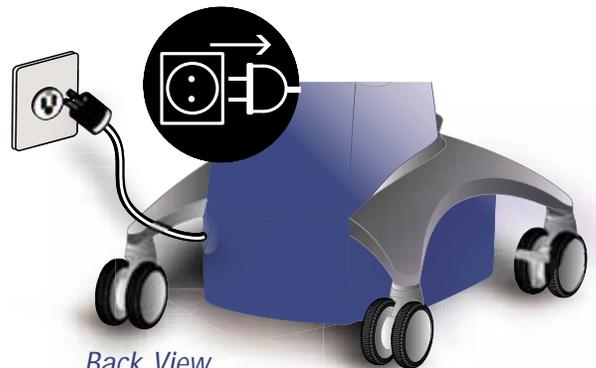


c Turn power system off by holding down the standby button  on the RUI for 1 - 3 seconds. Power light will shut off.

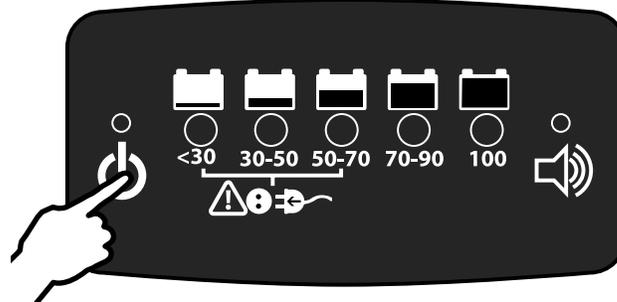


d Remove front cover and disconnect RUI cables from jacks located on side of outlet box.

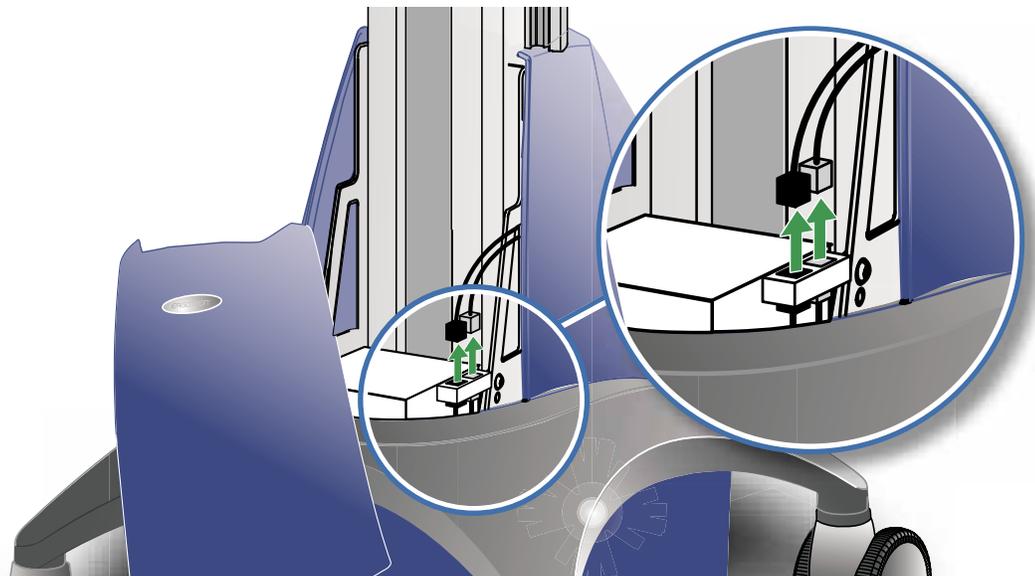


2 Remove battery cover.

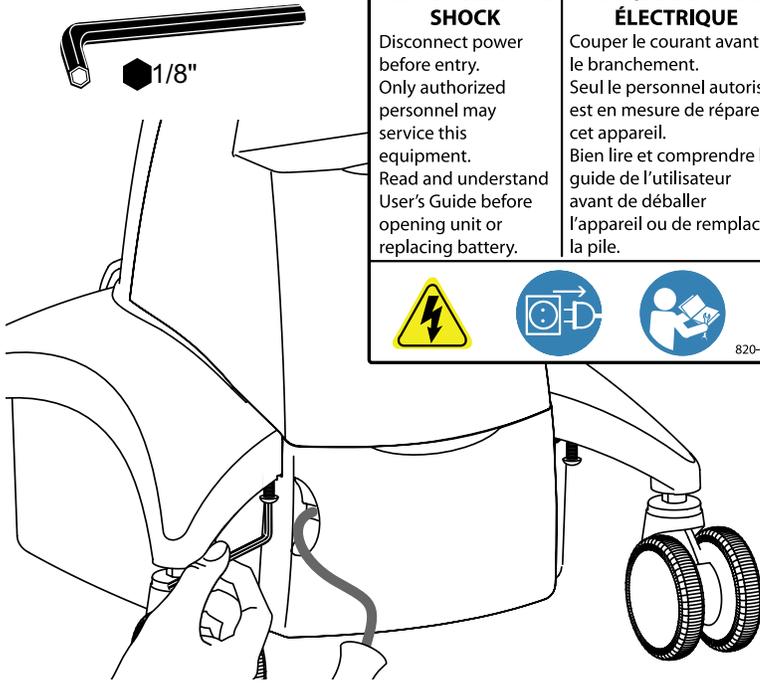

 WARNING AVERTISSEMENT	
<p>RISK OF ELECTRIC SHOCK</p> <p>Disconnect power before entry. Only authorized personnel may service this equipment. Read and understand User's Guide before opening unit or replacing battery.</p>	<p>RISQUE DE CHOC ÉLECTRIQUE</p> <p>Couper le courant avant le branchement. Seul le personnel autorisé est en mesure de réparer cet appareil. Bien lire et comprendre le guide de l'utilisateur avant de déballer l'appareil ou de remplacer la pile.</p>
	
820-599	

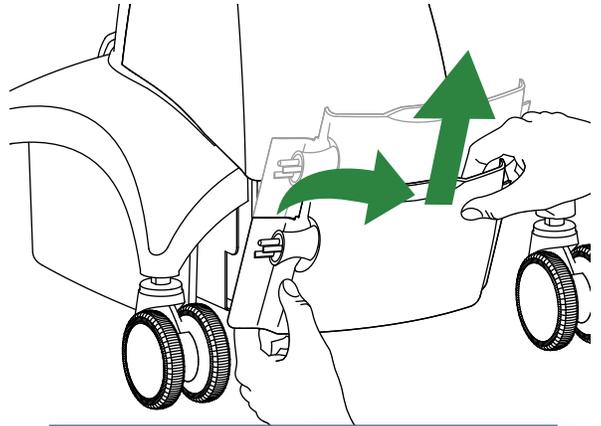







Unplug cord reel for better access.

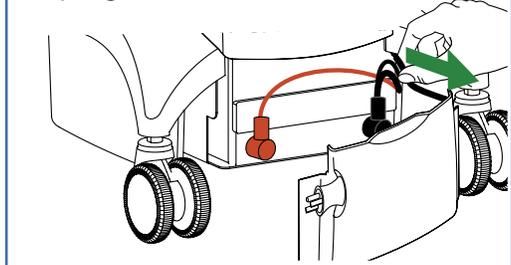

3 Remove protective boots, terminal bolts and cables.




Remove Black (negative) lead before removing Red (positive) lead. Be careful not to contact tools between terminals on battery and metal chassis.

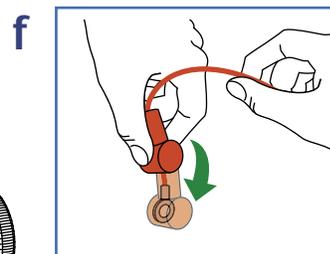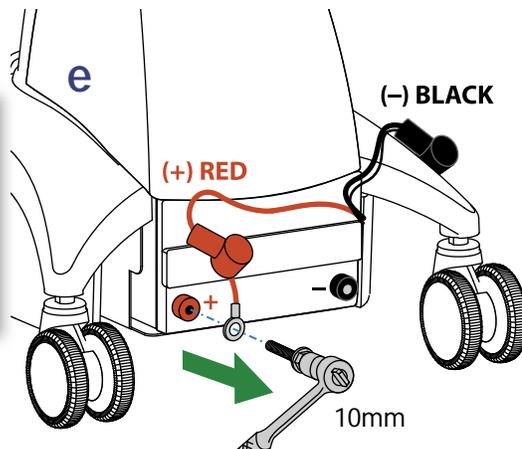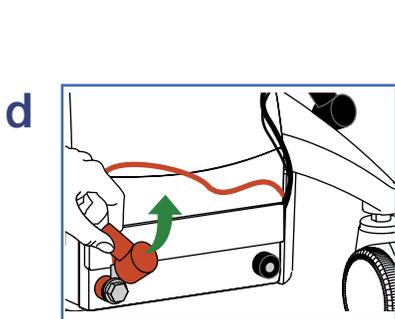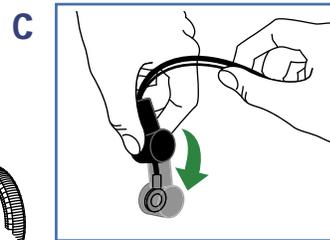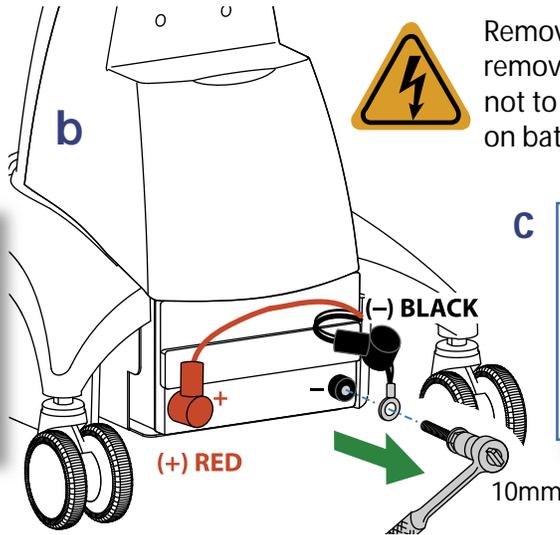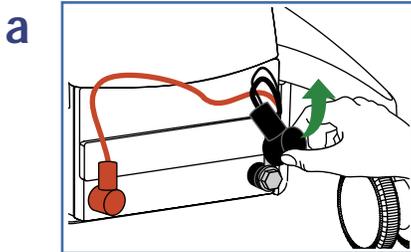


4 Remove old battery.

a

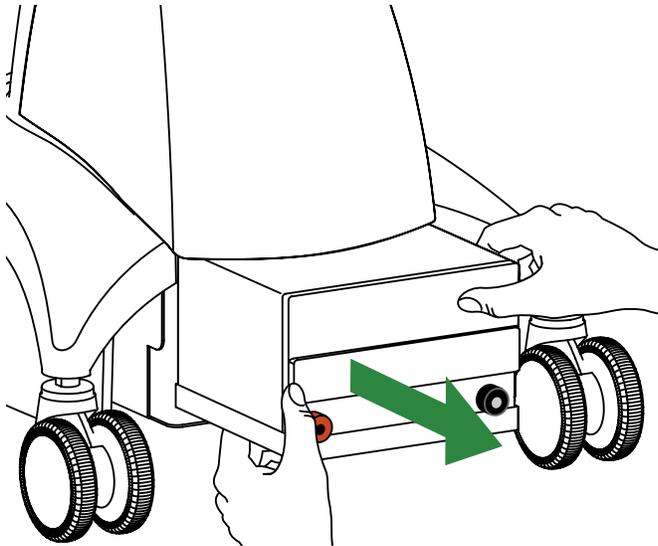


b

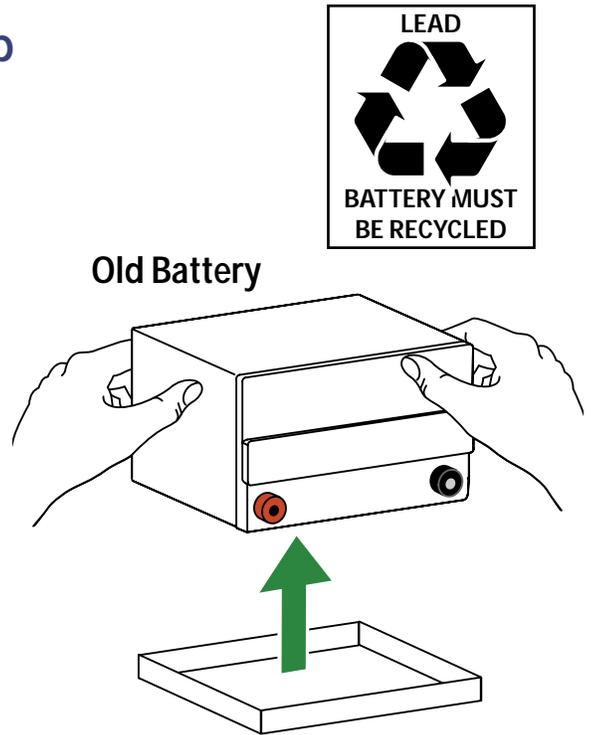


5 Place new battery in tray and install new battery.

a

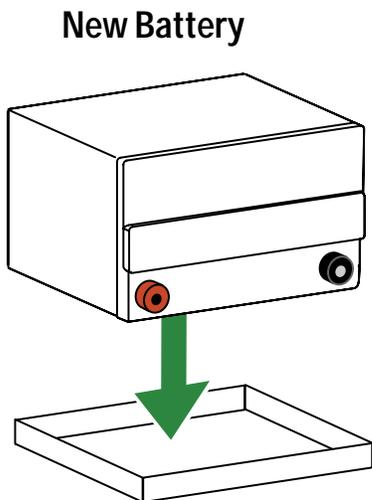


b

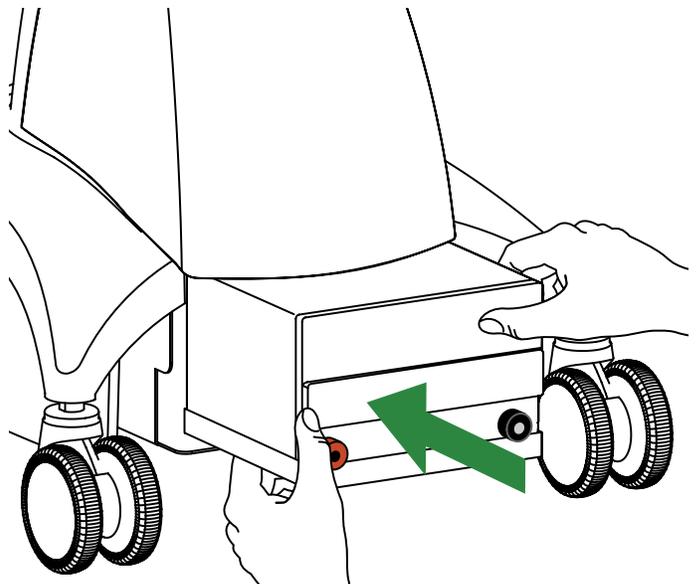




6 Reconnect cables with terminal bolts and replace protective boots.





Verify proper polarity when attaching cables. Failure to do so will result in blown fuse on battery harness.

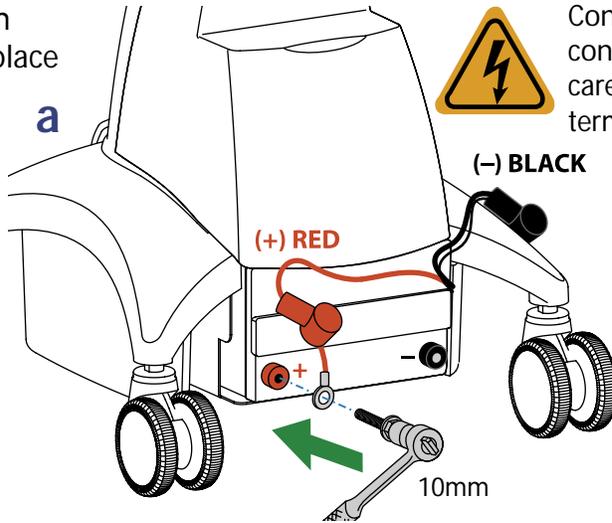

Connect Red (positive) lead before connecting Black (negative) lead. Be careful not to contact tools between terminals on battery and metal chassis.

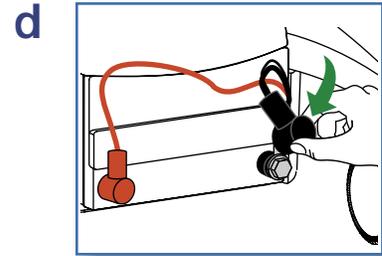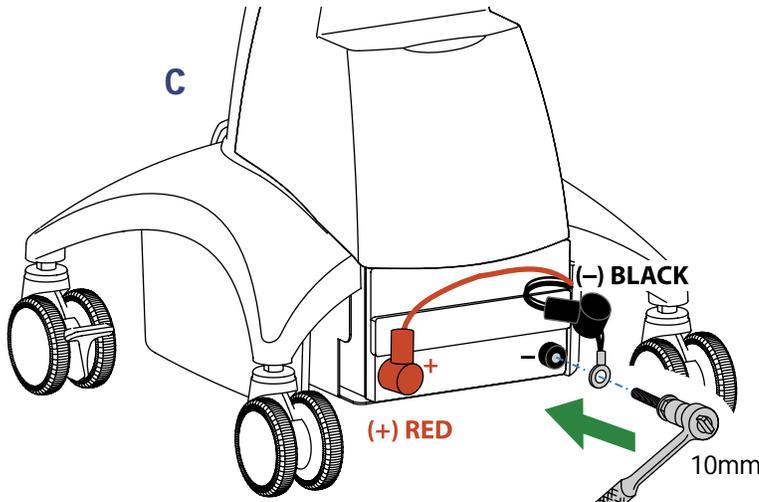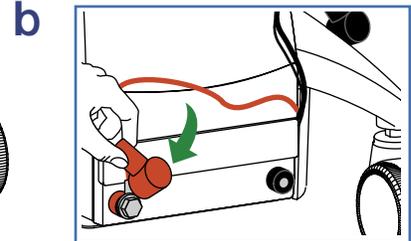


7 Replace battery cover, plugging the cord reel back in if unplugged, and tighten screws.

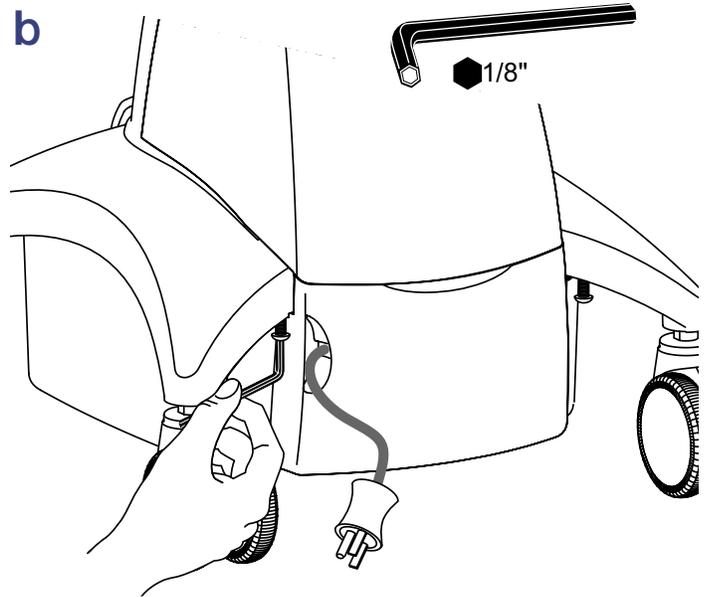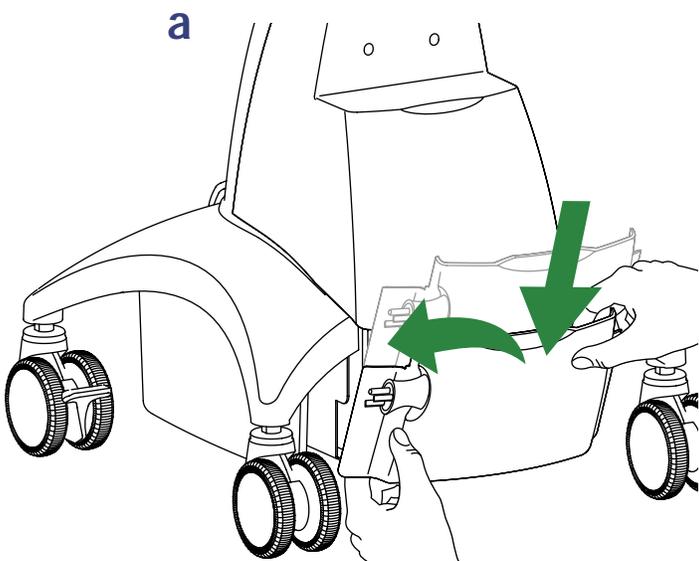


8 Initial Battery Charge

a Make sure RUI cables are connected.

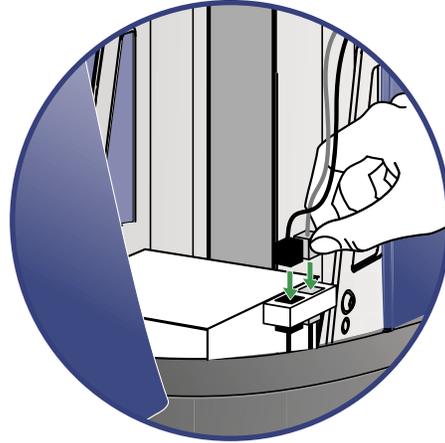


b Turn power system on by holding down the RUI Standby button  for 1 - 3 seconds. The Power light, the battery charge indicator lights and the alarm enabled light will come on. See page 6 for more details on RUI functions.

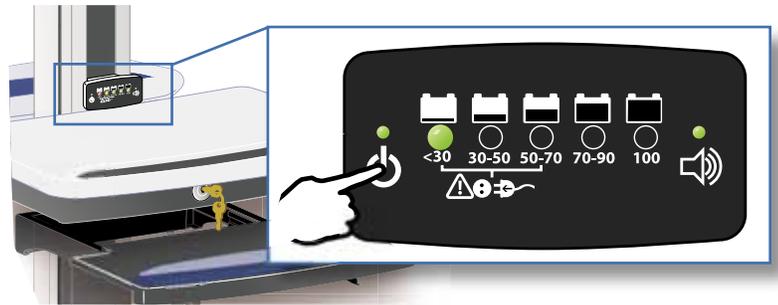


c Plug power cord into appropriate wall outlet.



CAUTION: The supplied Cord is rated for hospital use. Connecting the Cord to an outlet that is not hospital-grade (indicated with green dot) will not ensure grounding protection.

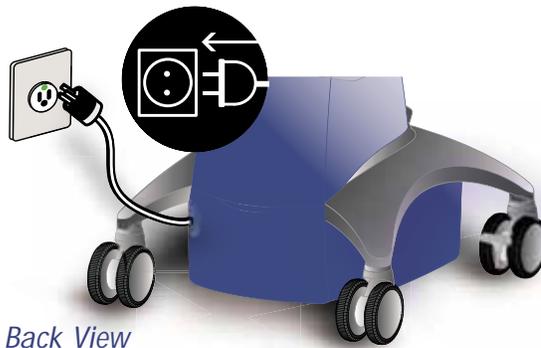

d Verify that the  light is flashing to indicate battery is charging. Continue to charge until light stops flashing and stays lit.

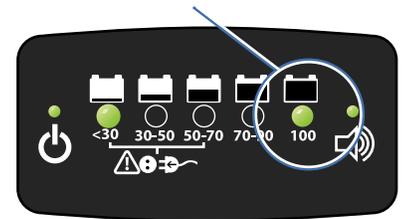

9 Ongoing Battery Charges

Charge battery as often as possible! With power system on, plug the power cord into an appropriate wall outlet. Plugging in the power cord for short periods of time DOES NOT affect the battery's ability to take a full charge, and prolongs battery life.

NOTE: Batteries may not achieve optimal performance until the 5th discharge / recharge cycle.

- Warranty on fully charged* batteries left in an unused state for more than three (3) consecutive months is automatically void.
- Warranty on fully discharged** batteries left in an unused state for more than three (3) consecutive days is automatically void. In order to avoid losing power to your equipment, Ergotron recommends that you charge the battery as often as possible and, if the system is not going to be used for 3 months or more, remove the battery.

*Fully charged: Battery charge will be at 100% and all RUI indicator lights will be illuminated.

**Fully discharged: Battery charge will be at 0% and all RUI indicator lights will be off.

Ergotron Warranty

Ergotron warrants that its products will remain free of defects in material and workmanship for five (5) years. All subject to the exceptions, exclusions and limitations as set forth below.

Exceptions to this warranty include:

- Three (3) year warranty on all Neo-Flex™ by Ergotron products
- One (1) year warranty on all power supply components by Ergotron; battery carries a 90 day performance warranty and six (6) month warranty against manufacturer defects.
- OEM products resold by Ergotron are warranted by the respective manufacturer

Exclusive Remedy: If any Ergotron equipment is or becomes defective in material or workmanship during the warranty period, Ergotron, at its sole discretion, may replace or repair the unit at a designated Ergotron location or at the customer location. The purchaser must get pre-approved authorization to return product to Ergotron. The Purchaser must contact Ergotron to obtain an RMA (Return Material Authorization) number. Ergotron will promptly repair or replace the item and ship it to Purchaser at no charge to the Purchaser. Performance of repair or replacement does not renew or extend the warranty period.

THIS REMEDY IS THE EXCLUSIVE REMEDY FOR BREACH OF WARRANTY AS TO THE EQUIPMENT, except in respect to the personal injury. Purchaser shall give Ergotron immediate written notice of any personal injury resulting from use of Ergotron equipment.

Exclusions and Limitations: Ergotron disclaims liability for injuries or losses caused by or associated with installation or use of its equipment in any manner other than in strict conformance with the instructions set forth in its installation manuals, supplemental assembly and installation instruction sheets, technical bulletins or product literature or by equipment that has been altered, damaged or misused.

Warranty on fully charge batteries left in an unused state for more than three (3) consecutive months is automatically void.


Warranty on fully discharged batteries left in an unused state for more than three (3) consecutive days is automatically void.

Purchaser has not relied upon and Ergotron disclaims any and all warranties other than the expressed warranty stated above, subject to the exclusions and limitations provided herein. Ergotron disclaims all other warranties, expressed or implied, including but not limited to the warranties of merchantability and fitness for a particular purpose.

Ergotron further disclaims liability to purchaser and all others for all consequential, incidental and special damages caused by or in connection with the products, their use or misuse, and their performance or nonperformance.

821-842-00

Contains electrical components. For information on disposal Contact Customer Service:

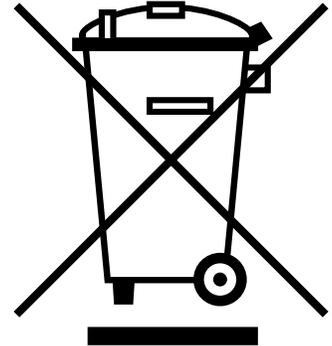

Product was placed on the market after 13 August 2005.

For Warranty information in other languages contact customer service.



USA 1-800-888-8458

Europe +31 (0)33-45 45 600

Ergotron, Inc.

World Headquarters
1181 Trapp Road, St. Paul, MN 55121 USA
Phone: 1-800-888-8458
Fax: 651-681-7717;
E-mail: sales@ergotron.com
Visit our website: www.ergotron.com

Ergotron Europe

Beeldschermweg 3
3821 AH Amersfoort
The Netherlands
Tel: +31 (0)33-45 45 600
Fax: +31 (0)33-45 649
Visit our website: www.ergotron-europe.com

Ergotron Canada

2000 Argentia Road Plaza IV, Suite 290
Mississauga, Ontario L5N 1W1, Canada
Phone: 1-800-267-9912
Fax: 1-905-567-4570;
E-mail: info.ca@ergotron.com



© 2006 Ergotron, Inc.
All rights reserved.

While Ergotron, Inc. makes every effort to provide accurate and complete information on the installation and use of its products, it will not be held liable for any editorial errors or omissions (including those made in the process of translation from English to another language), or for incidental, special or consequential damages of any nature resulting from furnishing this instruction and performance of equipment in connection with this instruction. Ergotron, Inc. reserves the right to make changes in the product design and/or product documentation without notification to its users. For the most current product information, or to know if this document is available in languages other than those herein, please contact Ergotron Customer Care at: 800-888-8458 or consult the Ergotron, Inc. website at: www.ergotron.com. No part of this publication may be reproduced, stored in a retrieval system, or transmitted in any form or by any means, electronic, mechanical, photocopying, recording or otherwise without the prior written consent of Ergotron, Inc., 1181 Trapp Road, Eagan, Minnesota, 55121, USA Patents Pending and Patented U.S. & Foreign. Ergotron is a registered trademark of Ergotron, Inc.