



StyleView Transfer Cart

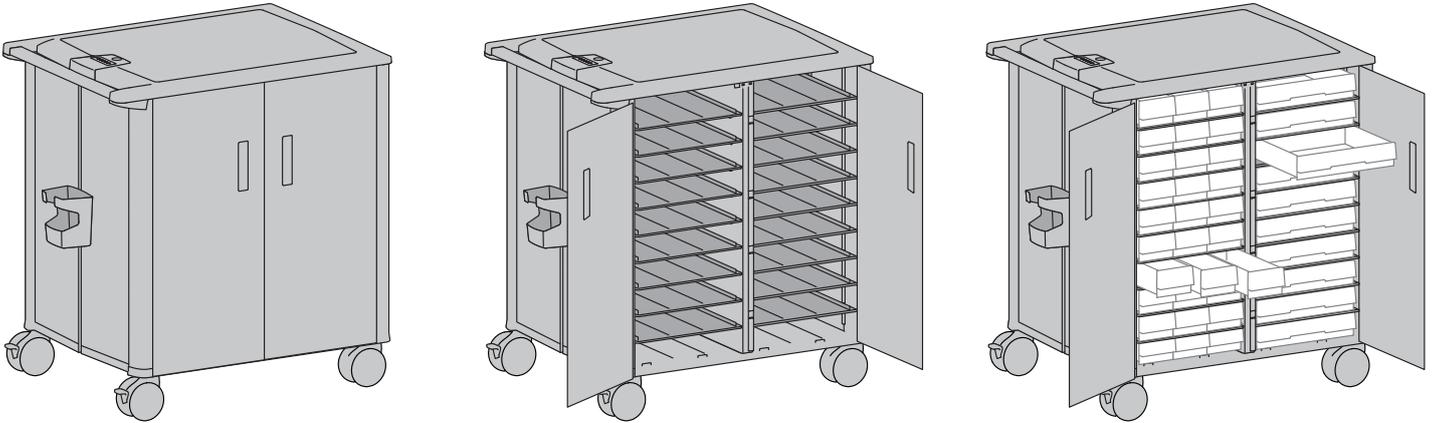


Drawers sold separately.

This cart is compatible with SV41, SV42, SV43, and SV44 drawers.
It is not compatible with SV31 or SV32 drawers.


Components

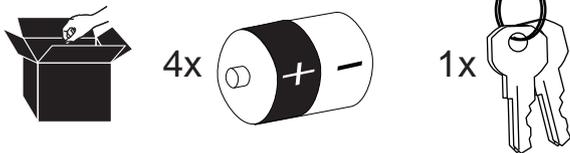

Tools Needed

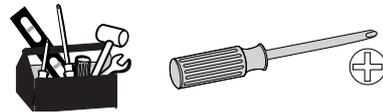

After unpacking the cart, remove the 4 protective layers from the ends of the cart.

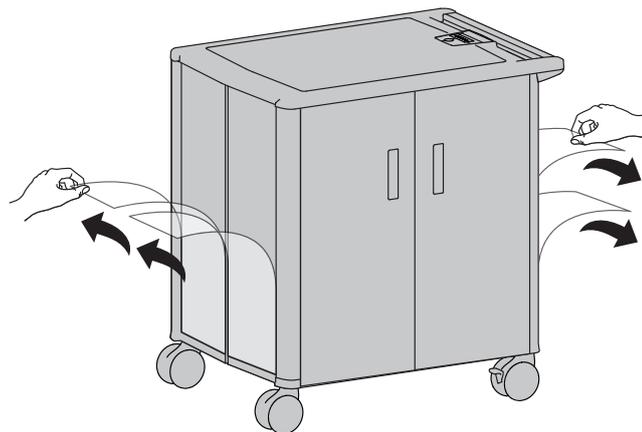

For the latest User Installation Guide please visit: www.ergotron.com




User's Guide - English
 Guía del usuario - Español
 Manuel de l'utilisateur - Français
 Gebrauchsgids - Deutsch
 Benutzerhandbuch - Nederlands
 Guida per l'utente - Italiano
 Användarhandbok - svenska
 ユーザーガイド : 日本語
 用户指南 : 汉语




www.ergotron.com
 USA 1-800-888-8458
 Europe +31 (0)33-45 45 600
 China 86-769-86018920


- 1** To install batteries for electronic lock function.
Unlock cart using provided key then lift right handle up to open door.

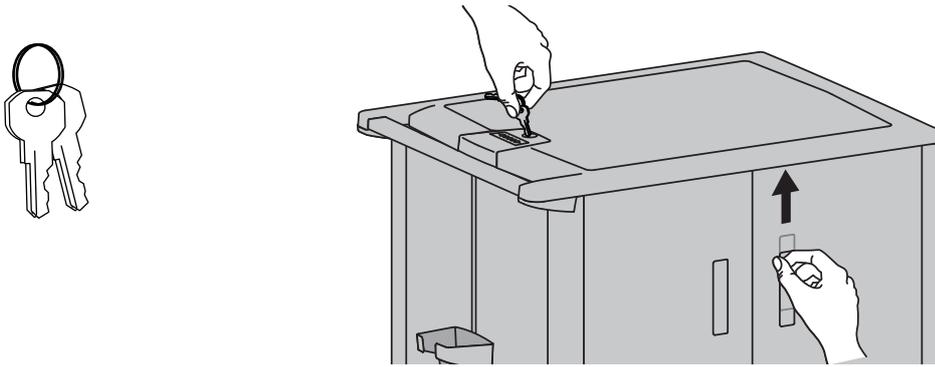


- 2** Open right door then left door.

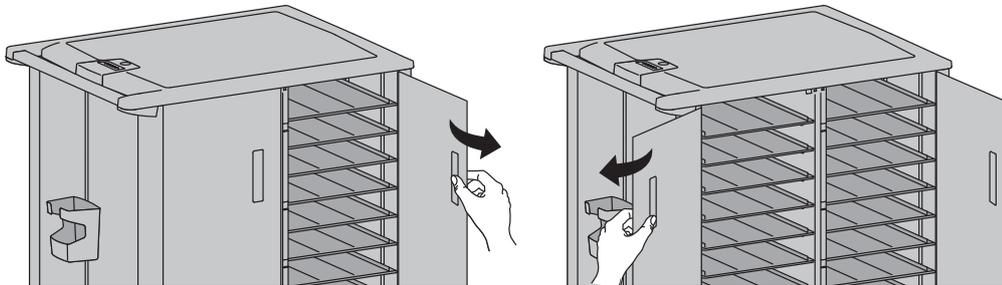

- 3** Push top cover tabs in and up to release top cover.


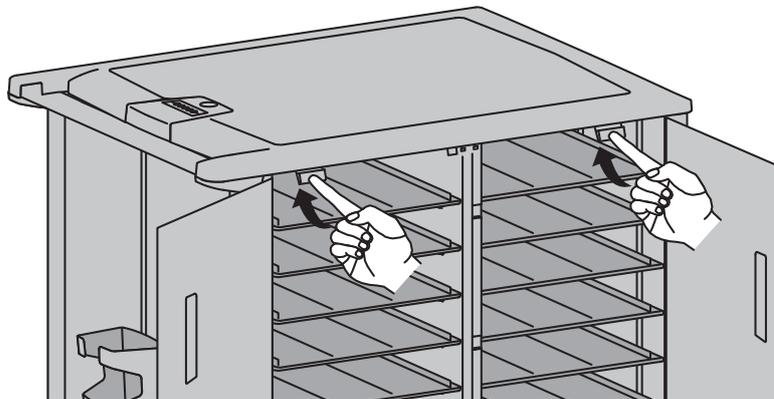


- 4** Lift front edge of top cover up a little then pull rear tabs out of slots to remove top cover.

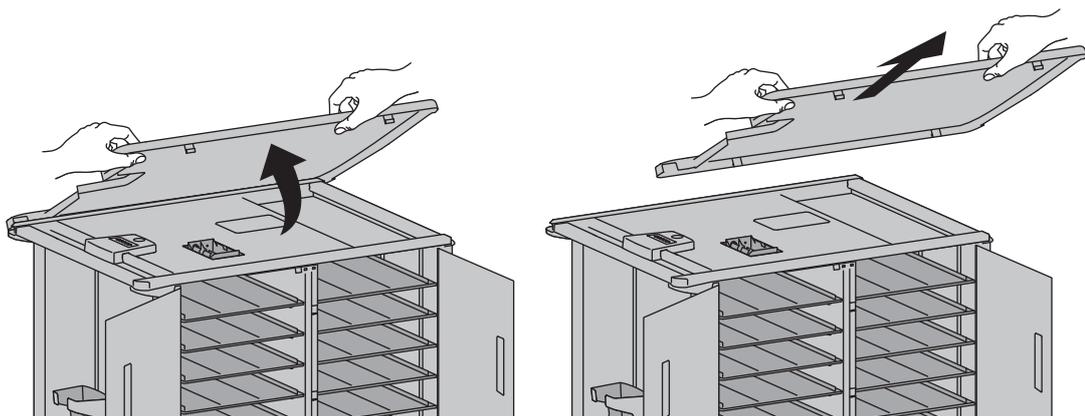


5

Install the 4 provided D batteries.

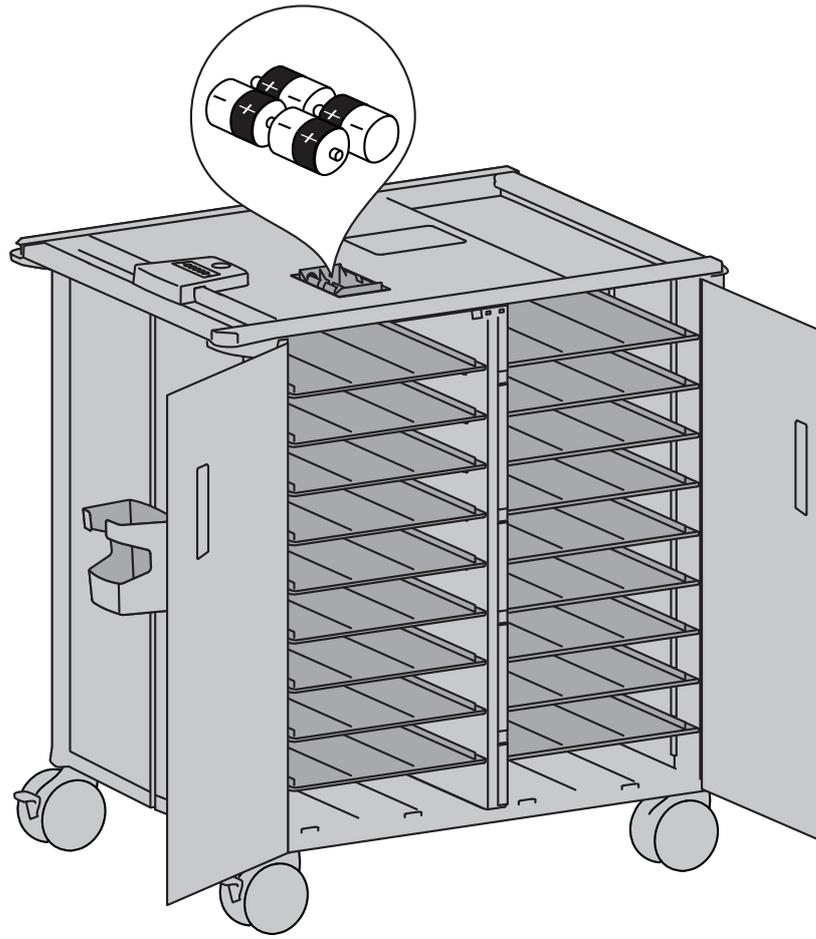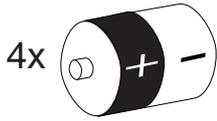

6





a. Slide top cover tabs into slots.
b. Lower cover so tabs slide into slots.
c. Make sure all 4 cover tabs click into place to secure top cover.

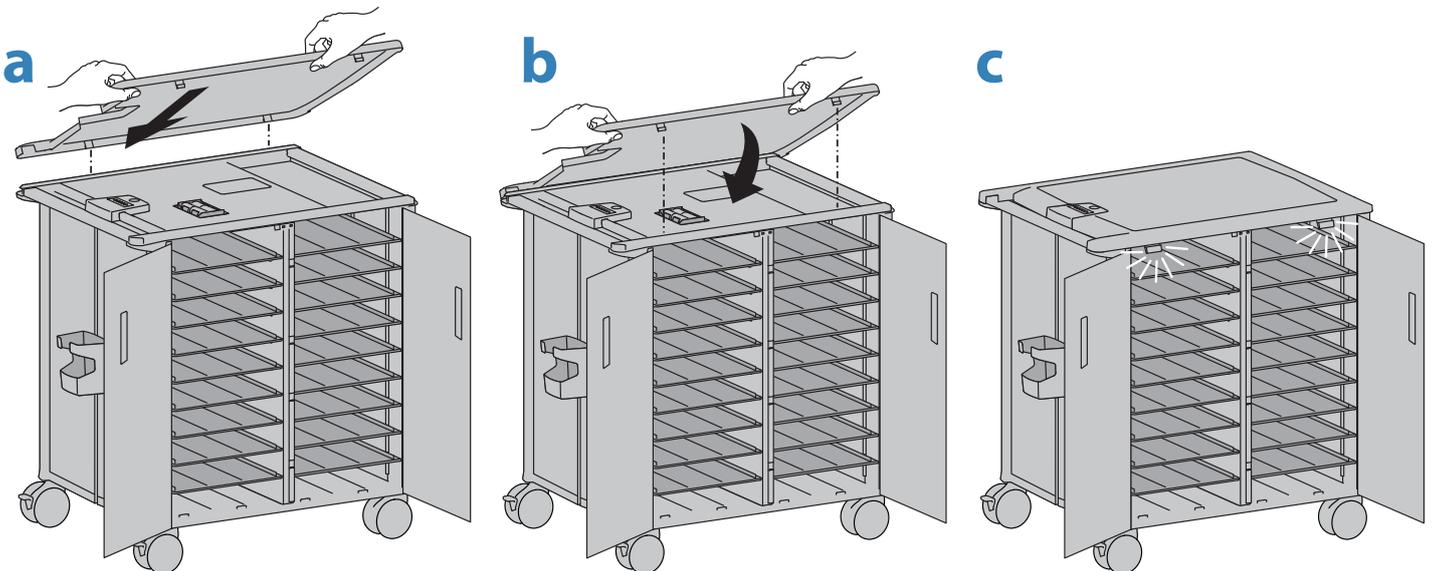

7 Install drawers by sliding onto shelves.





NOTE: Drawers sold separately.

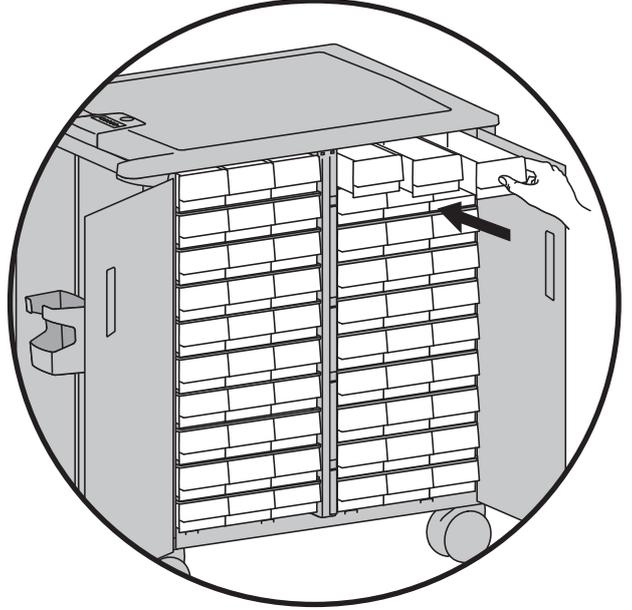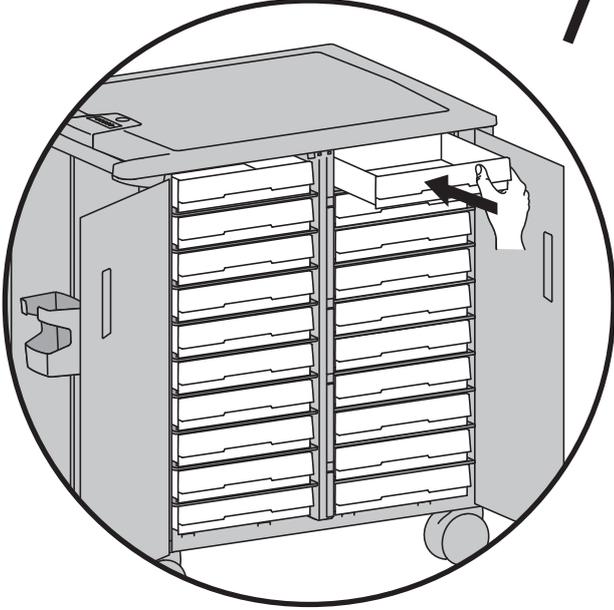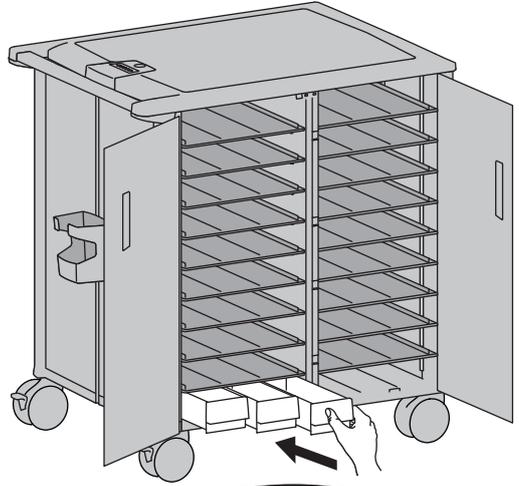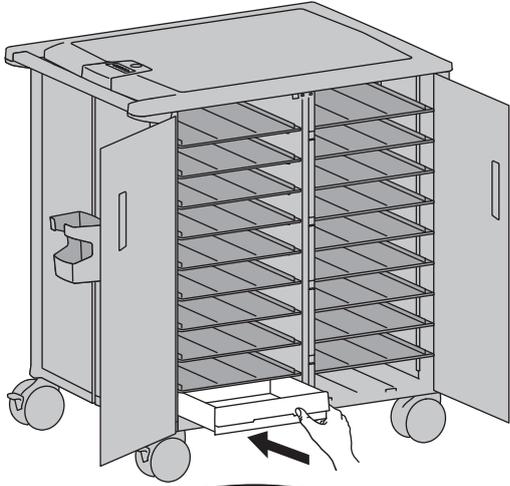

8

Open doors on other side of cart for shelf access.


- a. Unlock doors with key, then lift up on right handle to open door after unlocked.
- b. Slide drawers onto shelves.

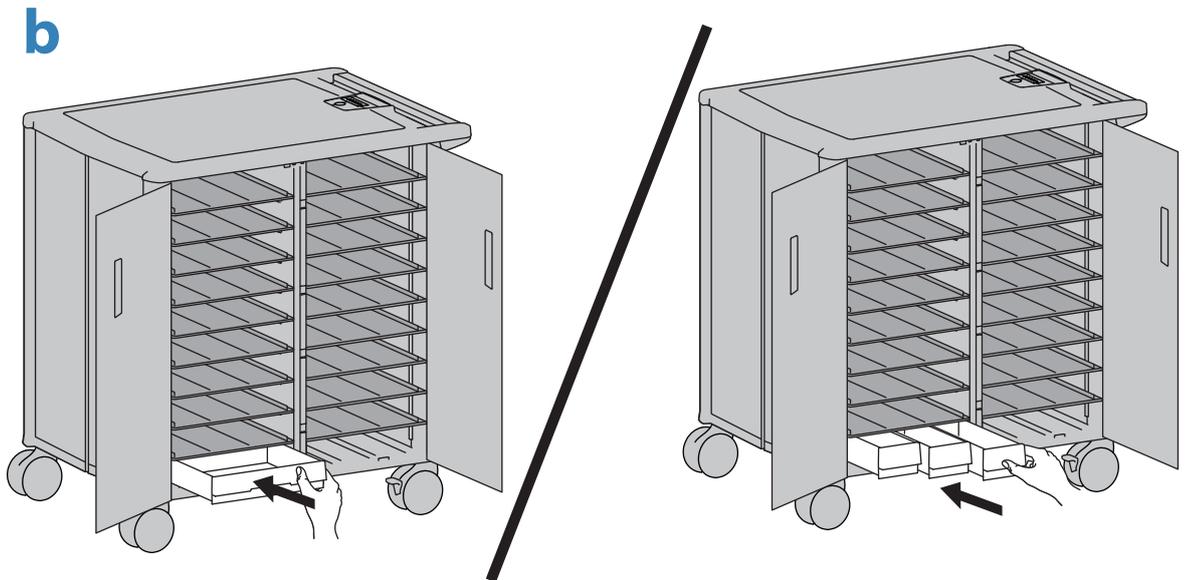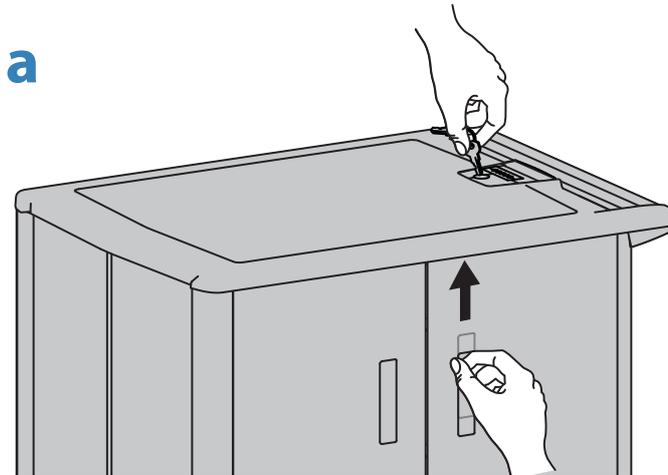

ENGLISH

How to adjust height of, or remove, scanner holder.


Do not remove screws! Loosen screws just enough to slide holder up or down in the T-slot. To remove holder, slide cup down and off the bottom of the cart.

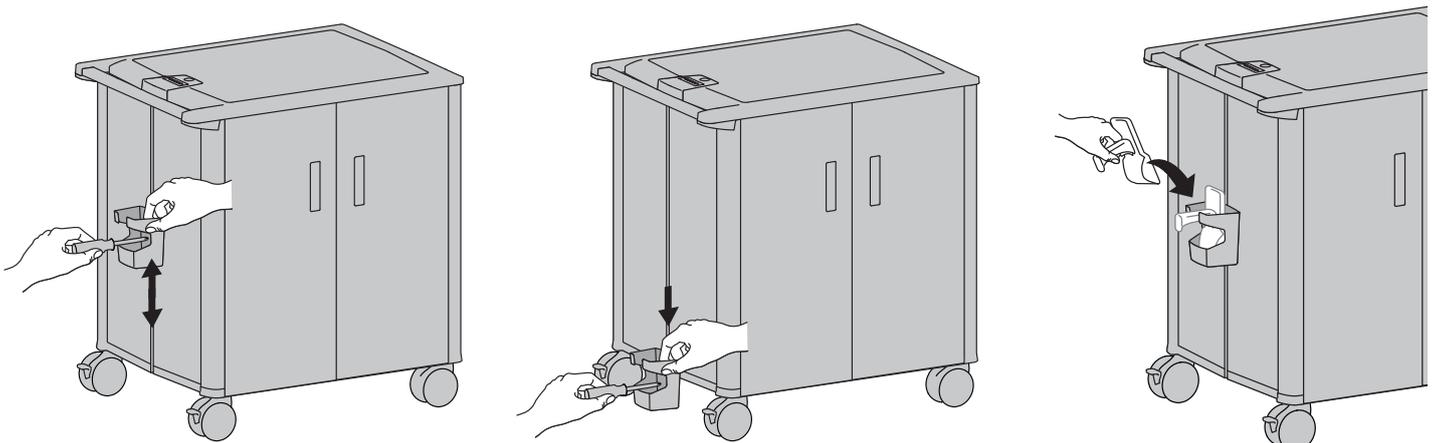

How to Use and Program Key Pad



NOTE: User should change Master PIN upon receipt of cart.

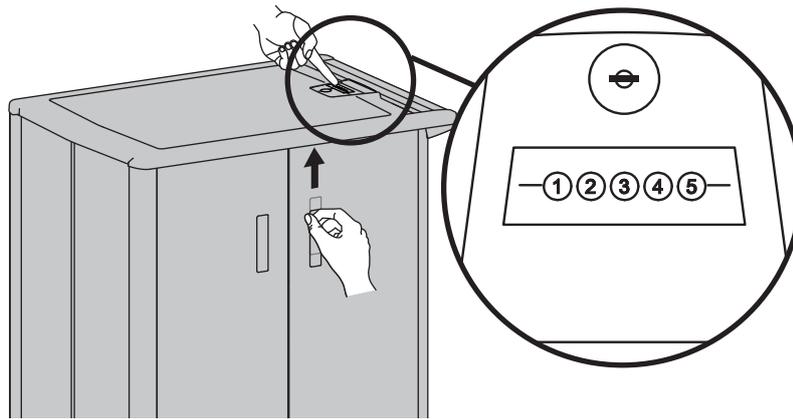

Programming New PIN (Default Master PIN: 12345)

- 1 Enter Master PIN.
- 2 Numbers 3 and 5 on the Keypad will start flashing, indicating the Master PIN has been accepted.
- 3 Simultaneously press 3 and 5 while they are flashing.
- 4 All the numbers will light (you have 3 seconds to enter a PIN).
- 5 Enter a new PIN—any sequence of 5 numbers. (The first PIN programmed is the new Master PIN.)
- 6 When the PIN is accepted, all of the LEDs will flash once in acknowledgement (you have 3 seconds to enter the next PIN).
- 7 You can then proceed to program up to 11 PINs by repeating steps 5 and 6.
- 8 When finished entering new PINs, system will exit program mode after 3 seconds with a series of LED flashes.

Unlock Door (3 methods):

- Enter Master PIN.
- Enter User PIN.
- Key - turn clockwise 1/4 turn

Lock Door (2 methods):

- Wait 4 seconds for door to lock automatically
- Press 1 and 2 on number pad simultaneously

Erase All Personal PINs and reset factory default Master PIN (cannot erase only one PIN):

- 1 Enter Master PIN.
- 2 Press 3 and 5 simultaneously (while blinking).
- 3 Press 1 and 2 simultaneously.

Lost Master PIN

Contact Ergotron Customer Care for instructions.

Drawer Troubleshooting

- Key pad numbers are dark.
 - Touch anywhere on number pad to activate back-light, (drawer remains locked until you enter valid PIN).
 - Check to make sure battery pack is connected.
- PIN is accepted but door does not open.
 - Make sure door is fully closed before entering PIN.
- #4 LED is flashing.
 - Replace batteries (D-cell alkaline only).
- Door won't open when User PIN is entered.
 - Test system by entering Master PIN. If door doesn't unlock, contact Ergotron customer care for "Lost Master PIN" instructions.

Flashing LED: Meaning


3 & 5: Unit is ready to program master or user PINs (see **Programming New PIN**)

All: New PIN accepted


4: Battery power is low

How to remove shelves.

- Push in and up on the 2 tabs under the shelf edge at the center of the cart.
- With shelf angled, slide out of cart.

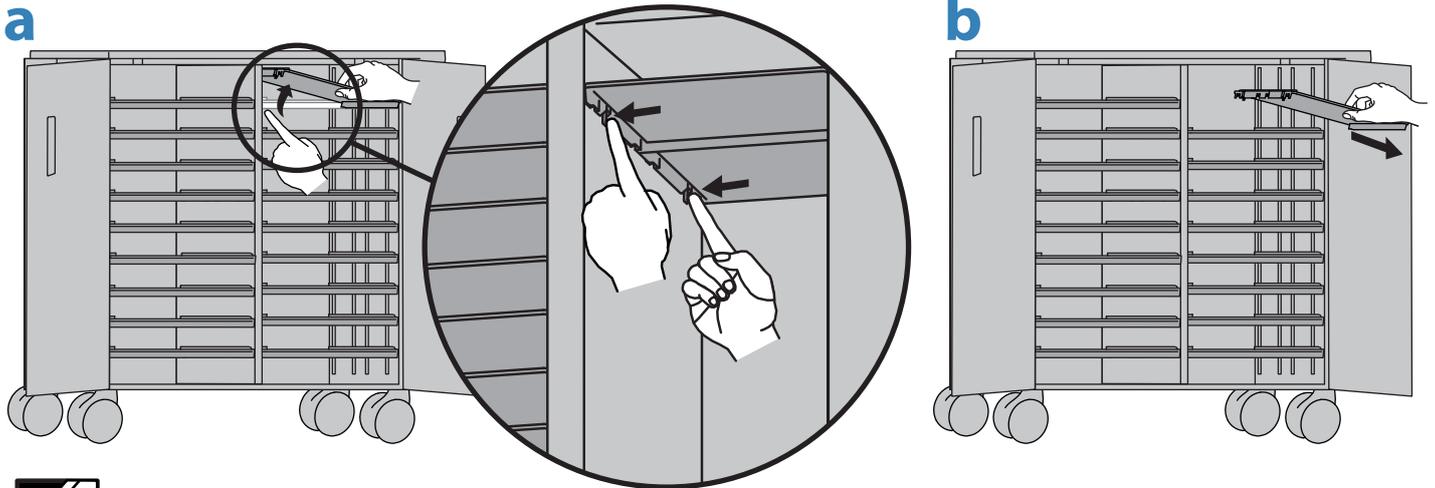


NOTE: Shelf tabs are at the center of the cart under each shelf.

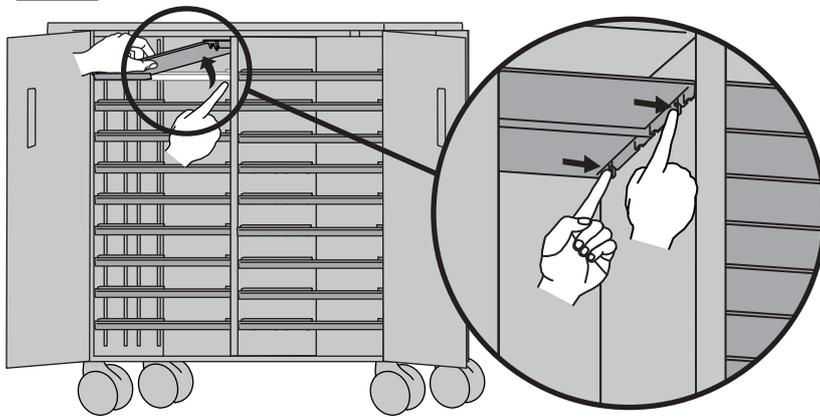

How to attach shelves.

- Slide shelf into desired slot at an angle.
- Lower center edge down onto shelf holders until tabs click into place.

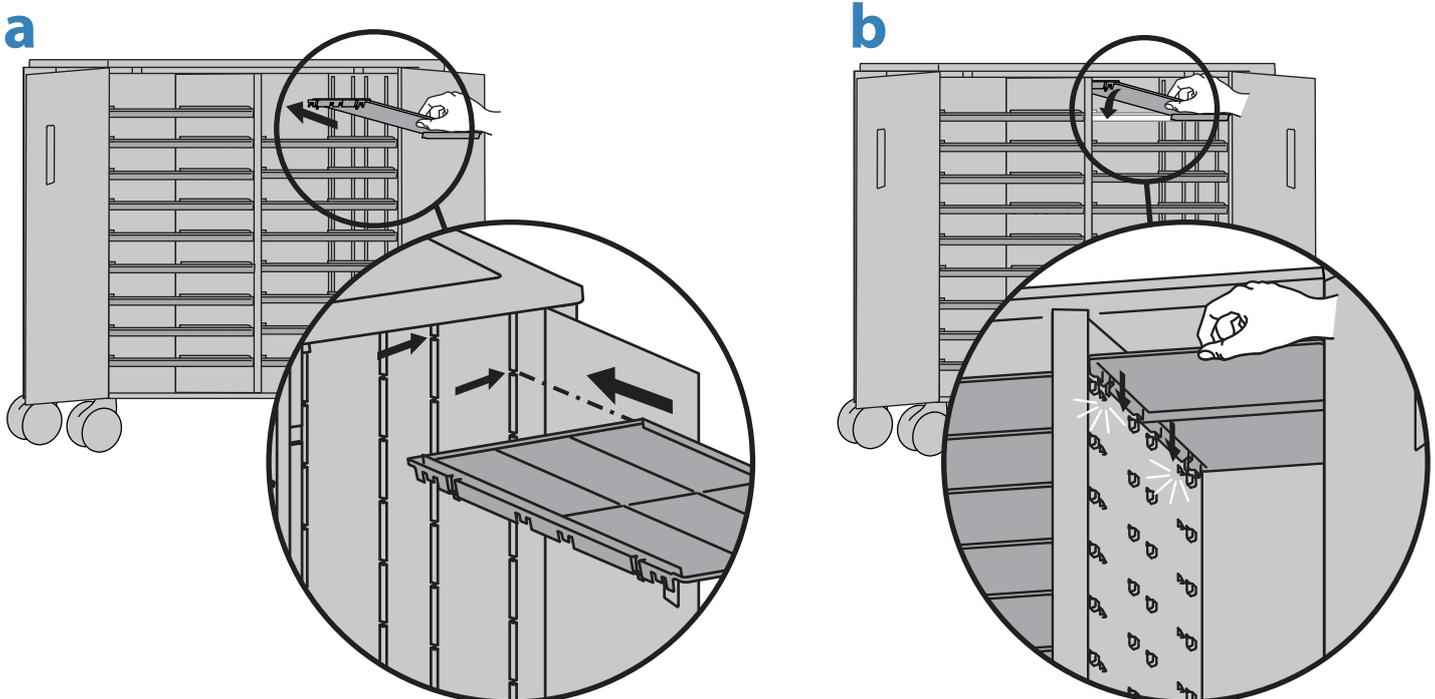

Maintenance & Safety

Cleaning and Maintenance

The following procedures are not guaranteed to control infection. The hospital infection control administrator or epidemiologist should be consulted regarding cleaning procedures and processes.

 To avoid risk of electric shock, do not expose electrical components to water, cleaning solutions or other potentially corrosive liquids or substances.

 Do not immerse Cart or Cart components in liquid or allow liquids to flow into the Cart. Wipe all cleaners off surface immediately using a damp cloth. Thoroughly dry surface after cleaning.

 Do not use flammable cleaners on Cart surfaces due to close proximity of electrical power and equipment.

All paints and plastic Cart components will withstand cleaning by most commonly used, diluted, non-abrasive solutions such as quaternary ammonia compounds, ammonia enzyme cleaners, bleach or alcohol solutions.

- Pen and permanent and dry erase markers can be removed with 91% isopropyl alcohol and a soft cloth.
- Iodine stains can be removed with commonly used cleaners and a soft cloth.
- Never use steel wool or other abrasive materials that will damage the surface finish.
- Do not use strong solvents such as trichloroethylene and acetone. These solvents will damage the surface finish.

It is recommended that any cleaning solution be tested on a small, inconspicuous area to ensure surface is not harmed.

 Casters - Check casters Monthly for wear and to make sure they are clean and free of debris that would prevent smooth travel. Avoid moving Cart across uneven, dirty or damaged surfaces.

 Please dispose of all batteries in accordance with local law.

 Always replace with similar size battery (D-cell) and always replace all 4 at the same time.

This Class A digital apparatus complies with Canadian ICES-003.

Cet appareil numérique de la classe A est conforme à la norme NMB-003 du Canada.

FCC Compliance Statement

The cart has been tested and found to comply with the limits for a Class A digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at his own expense.

Changes or modifications not expressly approved by Ergotron, Inc. could void the user's authority to operate the equipment.

Please contact Ergotron for complete EMC compatibility information.

この装置は、クラスA情報技術装置です。この装置を家庭環境で使用すると電波妨害を引き起こすことがあります。この場合には使用者が適切な対策を講ずるよう要求されることがあります。


VCCI-A

For service and warranty visit www.ergotron.com

For local customer care phone numbers visit: <http://contact.ergotron.com>



NOTE: When contacting customer service, reference the serial number.

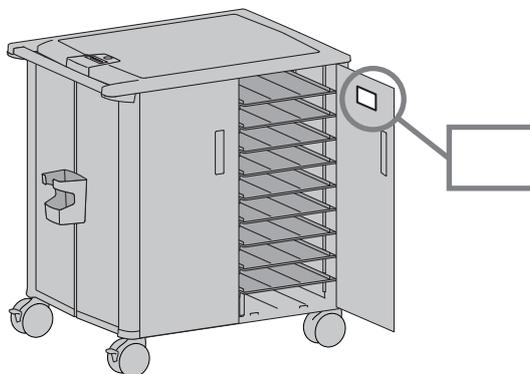



© 2013 Ergotron, Inc.
All rights reserved.

While Ergotron, Inc. makes every effort to provide accurate and complete information on the installation and use of its products, it will not be held liable for any editorial errors or omissions (including those made in the process of translation from English to another language), or for incidental, special or consequential damages of any nature resulting from furnishing this instruction and performance of equipment in connection with this instruction. Ergotron, Inc. reserves the right to make changes in the product design and/or product documentation without notification to its users. For the most current product information, or to know if this document is available in languages other than those herein, please contact Ergotron. No part of this publication may be reproduced, stored in a retrieval system, or transmitted in any form or by any means, electronic, mechanical, photocopying, recording or otherwise without the prior written consent of Ergotron, Inc., 1181 Trapp Road, Eagan, Minnesota, 55121, USA Patents Pending and Patented U.S. & Foreign. Ergotron is a registered trademark of Ergotron, Inc.

Americas Sales and Corporate Headquarters
St. Paul, MN USA
(800) 888-8458
+1-651-681-7600
www.ergotron.com
sales@ergotron.com

EMEA Sales

Amersfoort, The Netherlands
+31 33 45 45 600
www.ergotron.com
info.eu@ergotron.com

APAC Sales

Tokyo, Japan
www.ergotron.com
apacustomerservice@ergotron.com

Worldwide OEM Sales

www.ergotron.com
info.oem@ergotron.com