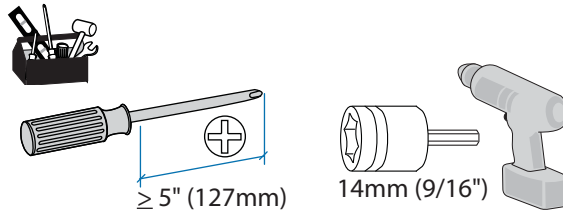
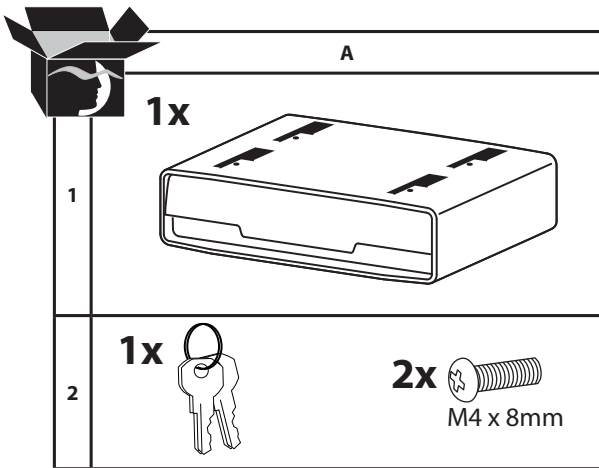
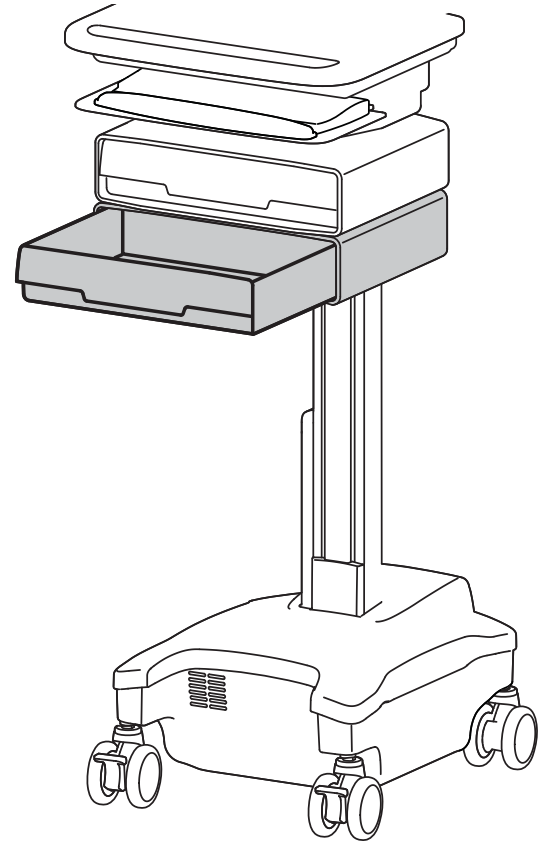
For service and warranty visit www.ergotron.com
 Si desea información de servicio y garantía, visite www.ergotron.com
 Pour toute question concernant la garantie ou le service client, consultez www.ergotron.com
 Weitere Informationen zu Service und Gewährleistung erhalten Sie unter www.ergotron.com.
 Ga voor service en garantie naar www.ergotron.com
 Per assistenza e la garanzia visitare www.ergotron.com
 Gå in på www.ergotron.com för service och garanti
 サービスおよび保証について詳しくは、www.ergotron.com をご覧ください。
 有关服务和保修，请访问 www.ergotron.com

11.8"
(300 mm)
14.6"
(370 mm)
3.4"
(86 mm)

≤ 4 lbs (1.8 kg) (without insert) ≤ 2 lbs (.8 kg) (with insert)
 ≤ 4 lbs (1.8 kg) (sin soporte de inserción) ≤ 2 lbs (.8 kg) (con soporte de inserción)
 ≤ 4 lbs (1.8 kg) (sans insert) ≤ 2 lbs (.8 kg) (avec insert)
 ≤ 4 lbs (1.8 kg) (ohne Einschub) ≤ 2 lbs (.8 kg) (mit Einschub)
 ≤ 4 lbs (1.8 kg) (zonder inzetstuk) ≤ 2 lbs (.8 kg) (met inzetstuk)
 ≤ 4 lbs (1.8 kg) (senza inserto) ≤ 2 lbs (.8 kg) (con inserto)
 ≤ 4 lbs (1.8 kg) 挿入部なし ≤ 2 lbs (.8 kg) 挿入部付き
 ≤ 4 lbs (1.8 kg) 不含插件 ≤ 2 lbs (.8 kg) 含插件
 ≤ 4 lbs (1.8 kg) ≤ 2 lbs (.8 kg)

Drawer Insert Weight = 2 lbs (.8 kg)
Peso del soporte de inserción = 900 gramos (2 libras)
Poids des inserts du tiroir = 2 livres (0,9 kg)
Gewicht des Einschubs = 2 lbs (.8 kg)
Gewicht inzetstuk lade = 2 lbs (0,9 kg)
Peso inserto cassetto = 0,9 kg
引出し挿入部の重量 = 2 lbs (0.9 kg)
抽屉插件重量 = 0.9 kg (2 lbs)
= 0.9kg (2lbs)

! WARNING

IMPACT HAZARD!
MOVING PARTS CAN CRUSH AND CUT.

Minimize Lift Tension BEFORE:

- Removing Mounted Equipment.
- Shipping Cart
- Storing Cart

To Minimize Lift Tension

1. Lower worksurface to lowest position.
2. Turn adjustment nut at top of riser counterclockwise until it stops (Adjustment may require 40-60 revolutions).

Failure to heed this warning may result in serious personal injury or property damage!

For More information and instructions refer to product guide at <http://4support.ergotron.com> or contact Ergotron Customer Care at 1-800-888-8458.

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
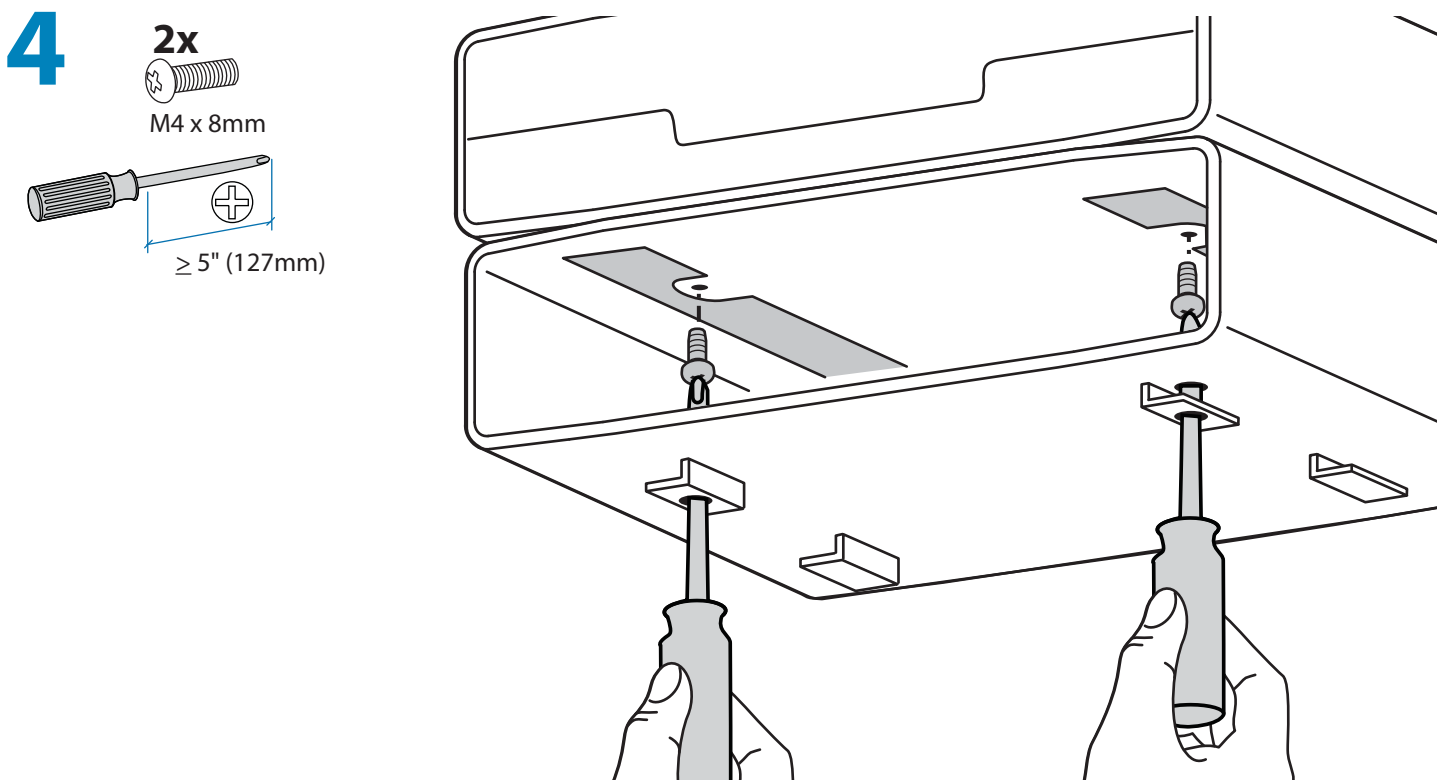
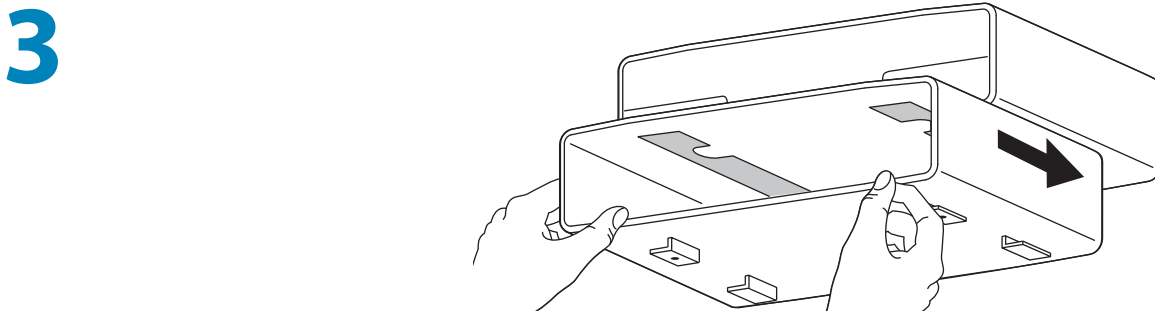
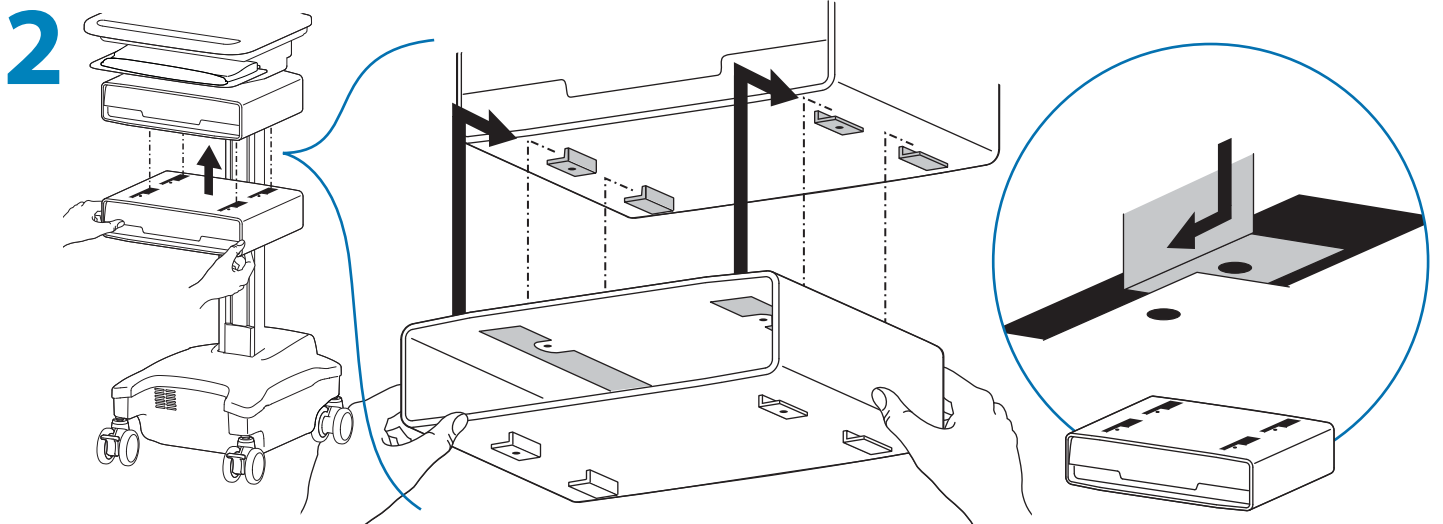
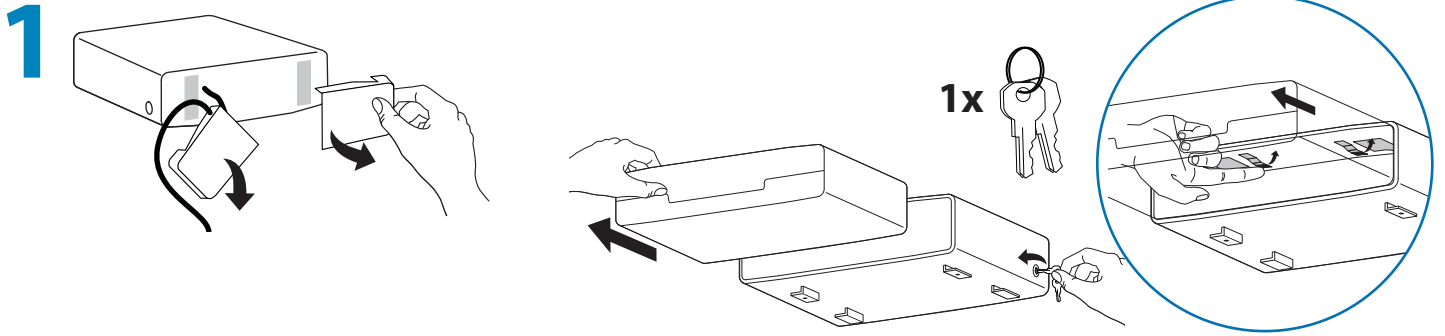
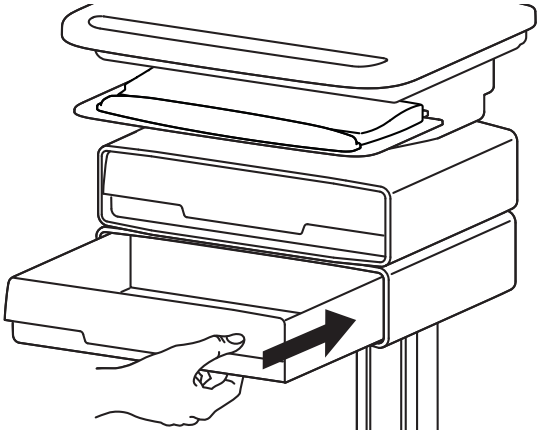
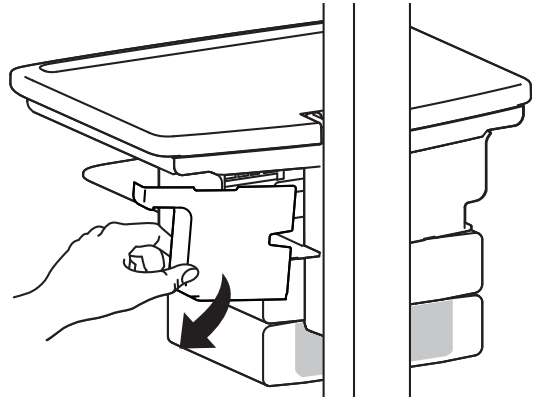
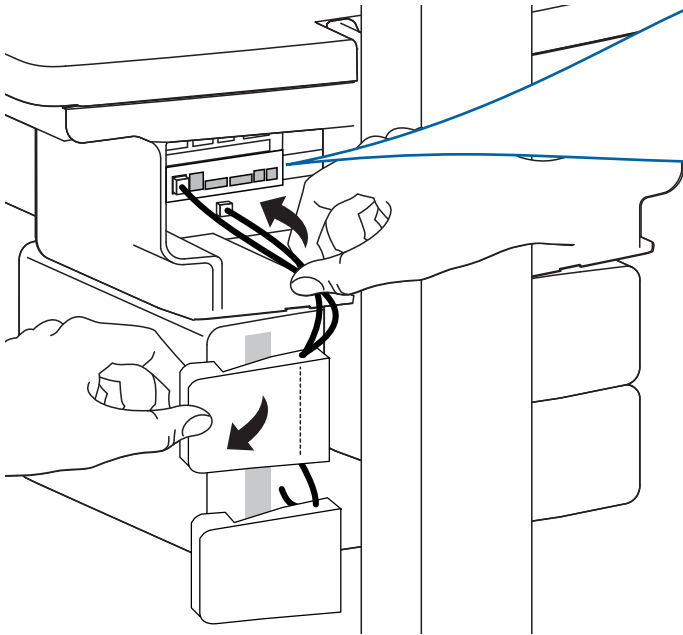
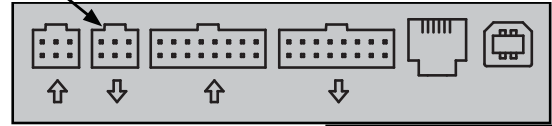
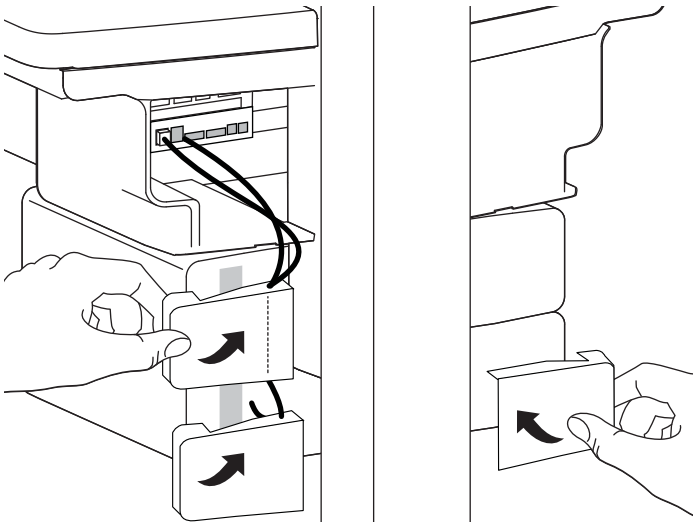
e

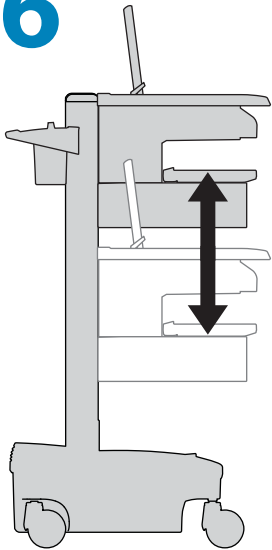6

It is important that you adjust this product according to the weight of the mounted equipment as described in the following steps. Any time equipment is added or removed from this product, resulting in a change in the weight of the mounted load, you should repeat these adjustment steps to ensure safe and optimum operation.

Adjustments should move smoothly and easily through the full range of motion and stay where you set it. If adjustments are difficult and do not stay in the desired position, follow the instructions to loosen or tighten the tension to create a smooth, easy adjustment motion. Depending on your product and the adjustment, it may take several turns to notice a difference.

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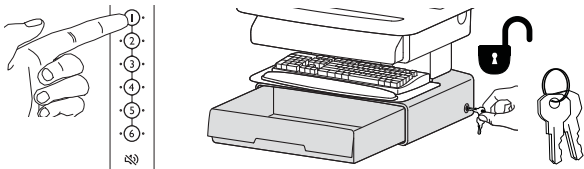
Lock Drawer:

- Wait 4 seconds for lock to engage automatically.
- NOTE: Drawer must be fully closed to lock.

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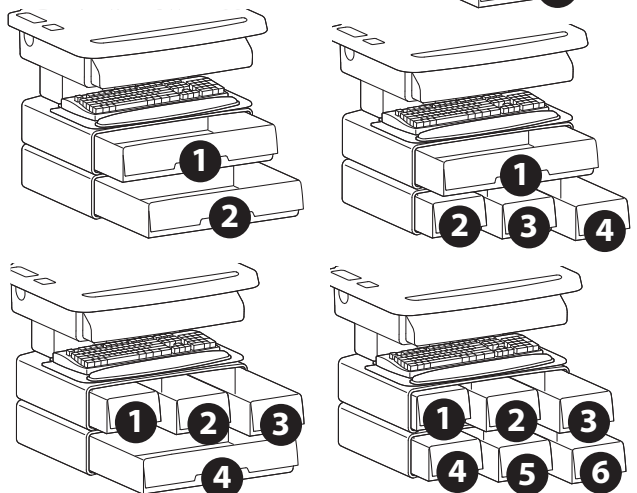
*Schubladennummern:

*Ladenummers:

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*The system is set up to allow only one drawer to be open at a time.

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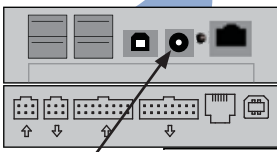
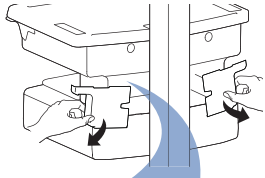
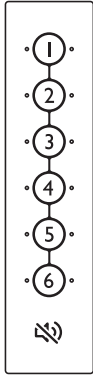
Drawer Troubleshooting

• Key pad numbers are dark:

- Touch anywhere on number pad to activate back-light, (drawer remains locked until you enter valid PIN).
- Check to make sure DC cable is connected.

• Drawer won't open when User PIN is entered:

- Test system by entering Master PIN. If drawer doesn't unlock, contact Ergotron customer care for "Lost Master PIN" instructions.

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- Provare a digitare il codice principale. If drawer doesn't unlock, contact Ergotron customer care for "Lost Master PIN" instructions.

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- 输入主密码测试系统。 If drawer doesn't unlock, contact Ergotron customer care for "Lost Master PIN" instructions.

Lights: Meaning
1,2,3,4,5 or 6 and Red mute button
Flashing and alarm sounding: Drawer is open longer than 20 seconds. Mute button can be pressed to mute alarm. System will not function until drawer is shut. The number corresponding to the open drawer will Flash Red on the keypad.
All available drawer numbers
Flashing Green: Waiting for drawer selection (see **Unlock Drawer**)
All available drawer numbers
Flashing Red: Firmware upgrade is happening
All flash once: New code accepted.
All flash three times: Exiting program mode.

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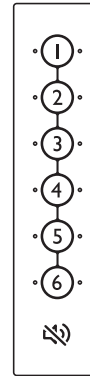
© 2010 Ergotron, Inc.
All rights reserved.

While Ergotron, Inc. makes every effort to provide accurate and complete information on the installation and use of its products, it will not be held liable for any editorial errors or omissions (including those made in the process of translation from English to another language), or for incidental, special or consequential damages of any nature resulting from furnishing this instruction and performance of equipment in connection with this instruction. Ergotron, Inc. reserves the right to make changes in the product design and/or product documentation without notification to its users. For the most current product information, or to know if this document is available in languages other than those herein, please contact Ergotron. No part of this publication may be reproduced, stored in a retrieval system, or transmitted in any form or by any means, electronic, mechanical, photocopying, recording or otherwise without the prior written consent of Ergotron, Inc., 1181 Trapp Road, Eagan, Minnesota, 55121, USA Patents Pending and Patented U.S. & Foreign. Ergotron is a registered trademark of Ergotron, Inc.