
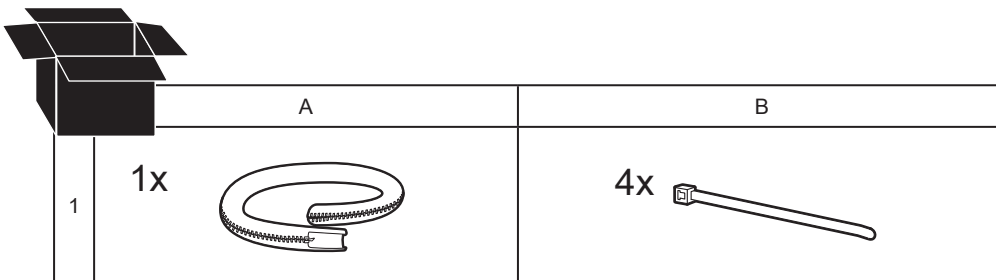
ergotron®

Cable Wrap Kit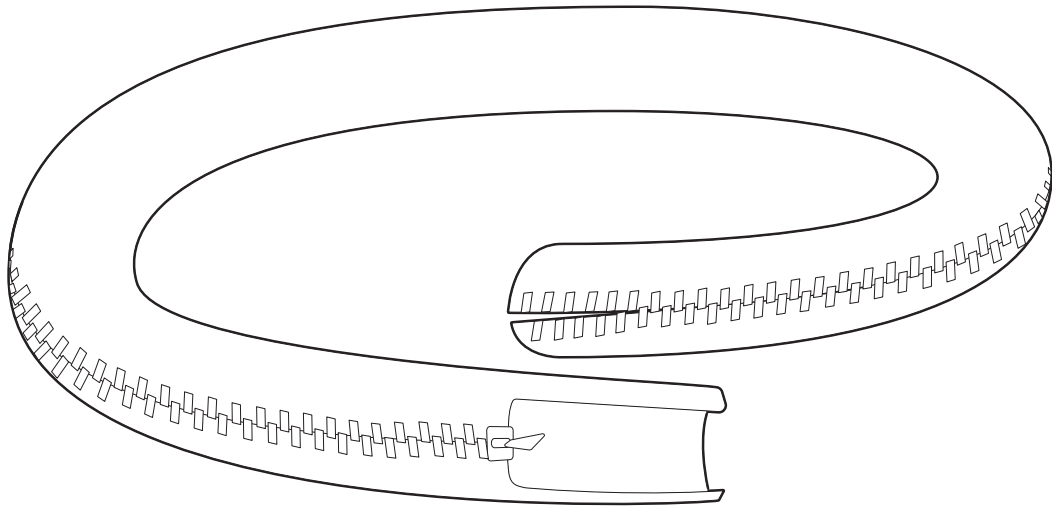

EN User Guide

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





www.ergotron.com | USA: 1-800-888-8458 | Europe: +31 (0)33-45 45 600 | China: 400-120-3051 | Japan: japansupport@ergotron.com

1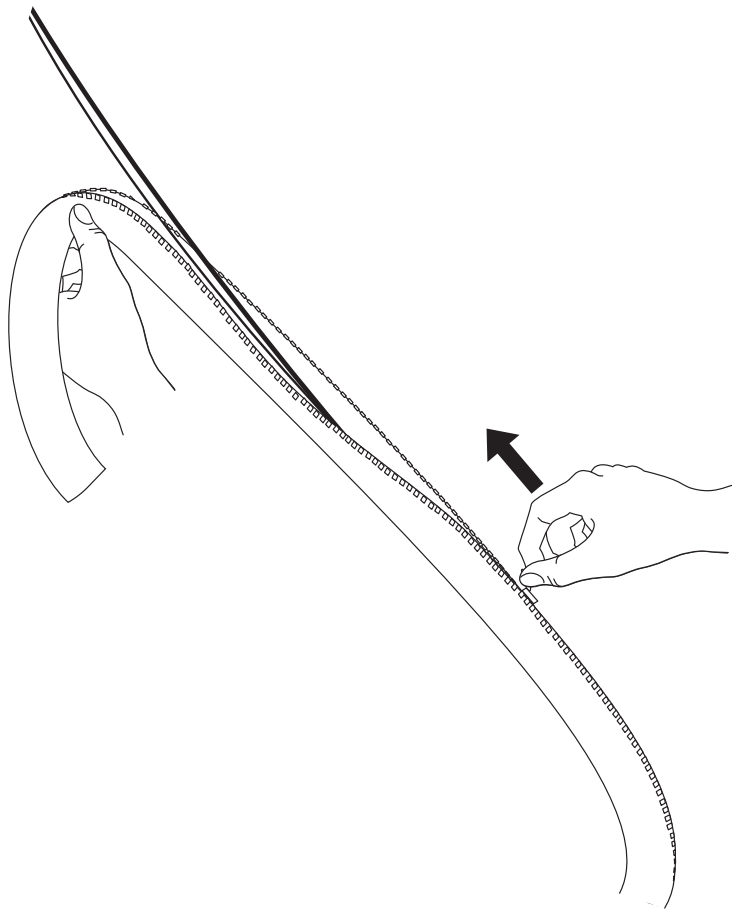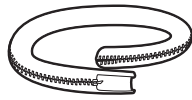

- EN** LifeKinnex Power Module attachment
- ES** Accesorio del módulo de alimentación LiFeKinnex
- FR** Fixation du module d'alimentation LiFeKinnex
- DE** Befestigung des LifeKinnex-Leistungsmoduls
- JA** LifeKinnex パワーモジュール付属品
- ZH** LifeKinnex 电源模块附件
- NL** Bijlage LifeKinnex-voedingsmodule
- IT** Collegamento del modulo di alimentazione LifeKinnex
- SV** Anslutning till LifeKinnex nätenhet

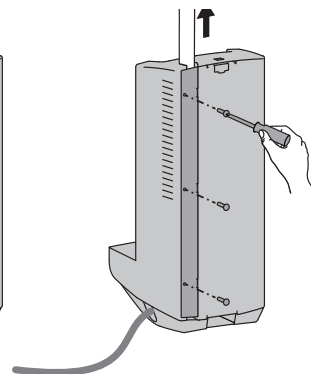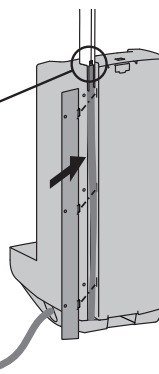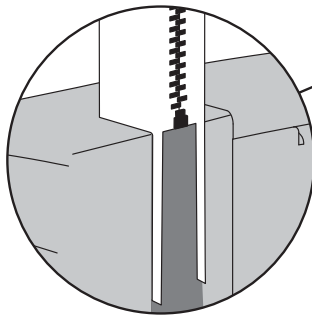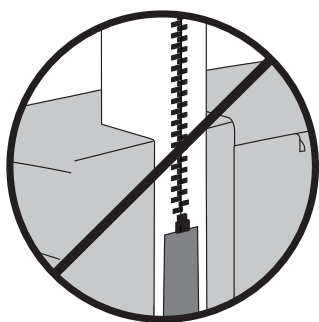
- EN** Attach Back Plate keeping Display and Power Strip cables in the Channel.
- ES** Conecte la placa trasera manteniendo los cables de la pantalla y la regleta de enchufes en el canal.
- FR** Fixez les câbles de l'écran et de la barre multiprise à l'arrière de la plaque arrière dans le canal.
- DE** Befestigen Sie die Rückwand. Achten Sie darauf, dass Display und Steckdosenleistenkabel im Kabelkanal bleiben.
- JA** ディスプレイとパワーストリップケーブルを経路に保持した背面プレートを取り付けます。
- ZH** 安装后面板，使显示器和电源板电缆始终在槽中。
- NL** Bevestig de achterwand en zorg ervoor dat de kabels van de display en de stekkerdoos in het kanaal blijven.
- IT** Collegare la piastra posteriore che mantiene i cavi del display e della presa multipla nella canalina.
- SV** Fäst bakstycket med display- och strömlistkablarna i kanalen.

2

1x

- EN** Secure cable wrap beneath head unit on CareFit Slim Carts
- ES** Asegure la brida para cables debajo de la unidad principal en los carros delgados CareFit
- FR** Fixer l'enveloppe de câble derrière l'unité de tête sur les chariots CareFit Slim
- DE** Sichern Sie das Kabelband unter der Kopfeinheit auf CareFit Slim Wagen
- JA** CareFitスリムカートのヘッドユニットの下にケーブルラップを固定します
- ZH** 将电缆包固定在 CareFit Slim 推车的头部下方
- NL** Maak een snoerwikkel vast onder de hoofdeenheid op CareFit Slim Carts
- IT** Fissare l'avvolgimento cavi sotto l'unità di testa sui Carrelli CareFit Slim
- SV** Fäst kabelomslaget under huvudenheten på CareFit Slim-vagnarna

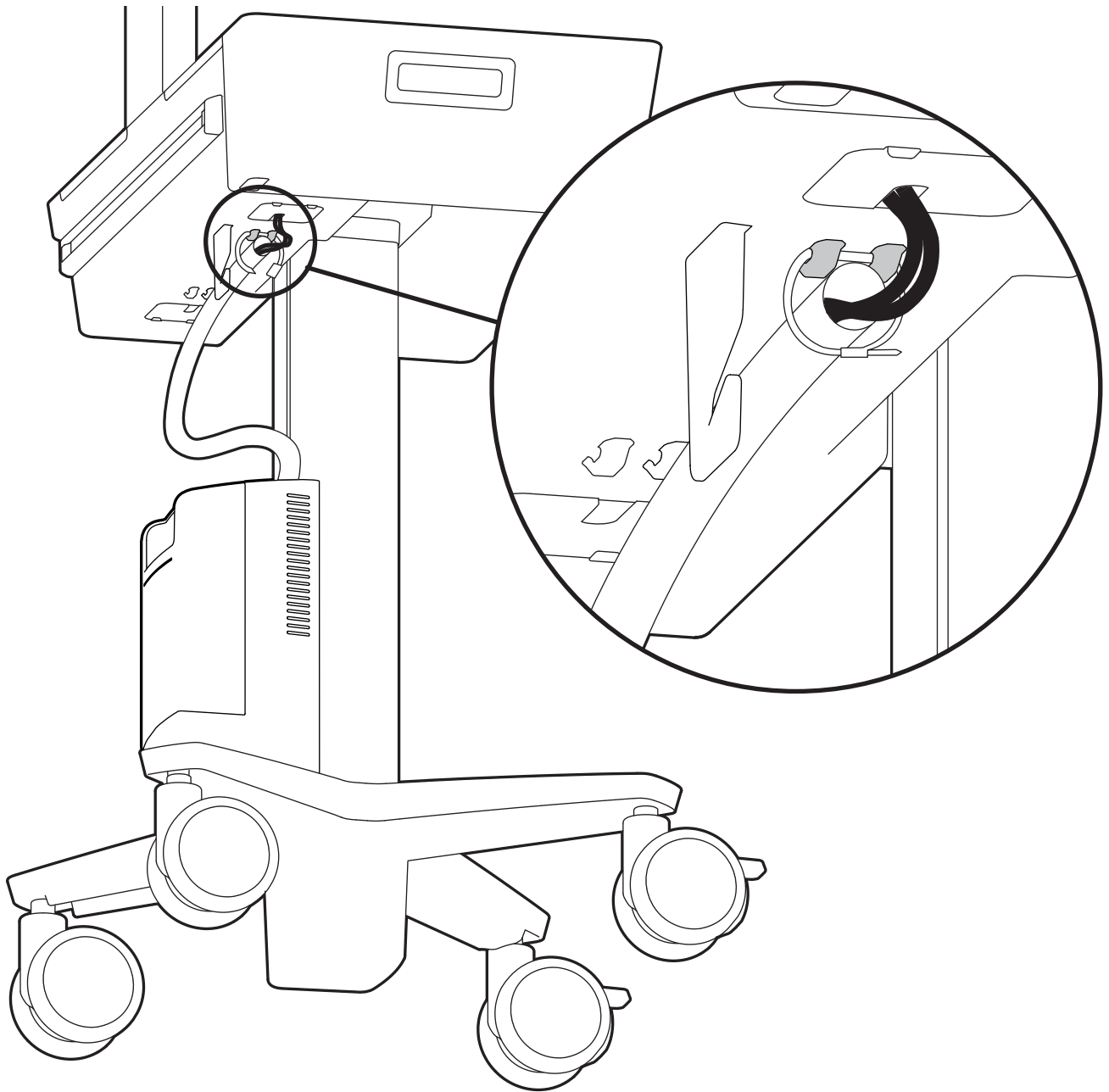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© 2019 Ergotron, Inc. All rights reserved.

While Ergotron, Inc. makes every effort to provide accurate and complete information on the installation and use of its products, it will not be held liable for any editorial errors or omissions (including those made in the process of translation from English to another language), or for incidental, special or consequential damages of any nature resulting from furnishing this instruction and performance of equipment in connection with this instruction. Ergotron, Inc. reserves the right to make changes in the product design and/or product documentation without notification to its users. For the most current product information, or to know if this document is available in languages other than those herein, please contact Ergotron. No part of this publication may be reproduced, stored in a retrieval system, or transmitted in any form or by any means, electronic, mechanical, photocopying, recording or otherwise without the prior written consent of Ergotron, Inc., 1181 Trapp Road, Eagan, Minnesota, 55121, USA Patents Pending and Patented U.S. & Foreign. Ergotron is a registered trademark of Ergotron, Inc.