


# CAREFIT™

by ergotron®

---

## Touch Screen Instructions

for CareFit Pro Electric Lift Monitor Cart  
Powered

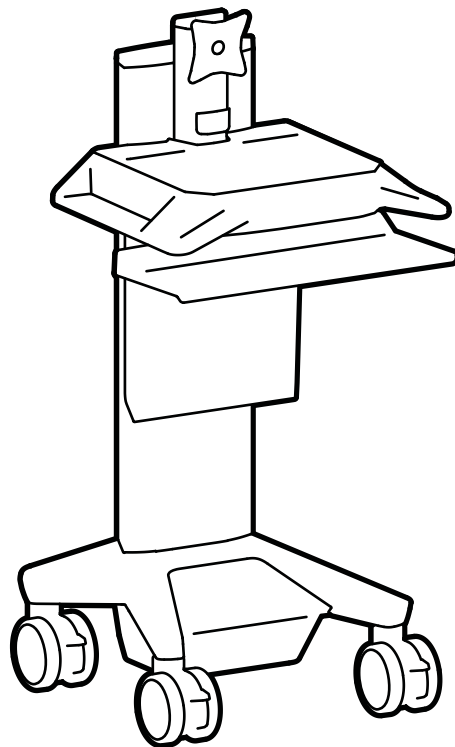

**English**




For the latest User Installation Guide please visit: [www.ergotron.com](http://www.ergotron.com)

English, Español, Français, Deutsch, Nederlands, Italiano, Svenska, 日本語, 汉语

[www.ergotron.com](http://www.ergotron.com) | USA: 1-800-888-8458 | Europe: +31 (0)33-45 45 600 | China: 400-120-3051 | Japan: [japansupport@ergotron.com](mailto:japansupport@ergotron.com)

# Touch Screen Control Panel

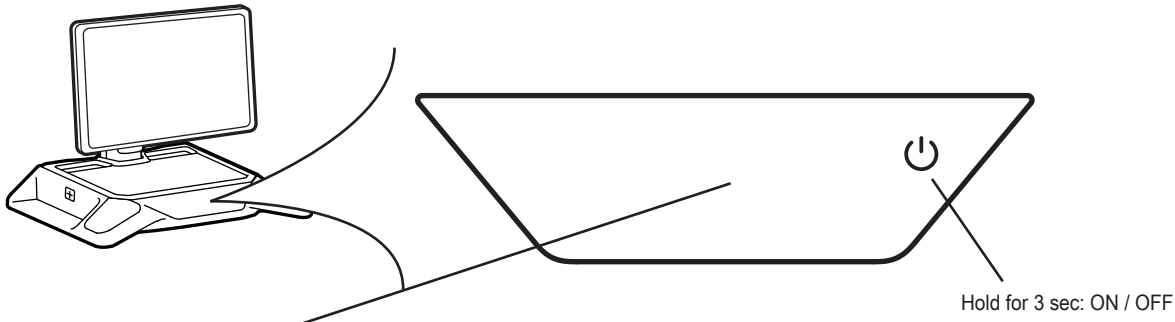

## HOME SCREEN

### Wi-Fi status

- Not Connected
- Connected but Not Configured
- Connected

Return to Home Screen

**Home Screen Elements:**

- Wi-Fi Status:** Located in the top left corner.
- Battery Charge Level:** A battery icon with a percentage (90%) and time remaining (8hr 2min).
- Clock:** Displays the current time (10:23 PM) and date (6:26 AM).
- Settings:** A gear icon in the top right corner.
- Navigation:** Up and down arrow buttons for raising and lowering the cart.
- Lights:** A lightbulb icon to toggle the cart lights.
- Charging Status:** A plug icon with a lightning bolt, indicating whether the cart is plugged in and charging or discharging.

**Menu Options:** A separate box lists the following options: DISPLAY BRIGHTNESS, SYSTEM INFORMATION, TIME CONFIGURATION, and ABOUT. A close button (X) is in the top right corner of this menu.

**Navigation:** A 'Return to Home Screen' button is located in the top right corner of the menu box.

Return to Home Screen

**Cart Lights Control Panel:**

- Back Arrow:** A circular button with a left-pointing arrow to return to the Home Screen.
- Light Status:** Shows '8h 2m' and a battery icon.
- Lightbulb Icon:** Labeled 'ALL', used to turn all lights on/off.
- Light Categories:** Three sections for 'TOP', 'KEYBOARD', and 'BASE' lights, each with a gear icon and a slider to adjust brightness.
- Dim/Bright Labels:** 'Dim' is on the left and 'Bright' is on the right of the sliders.






**Turns lights on/off:** Points to the lightbulb icon.

**Turns all lights on/off:** Points to the 'ALL' lightbulb icon.

**Cooling Down Warning:** An hourglass icon appears on the right side of the Home Screen, indicating that the lift engine has been moving too much and needs time to cool down.

This symbol means the lift engine has been moving too much and needs time to cool down.

# Touch Screen Control Panel

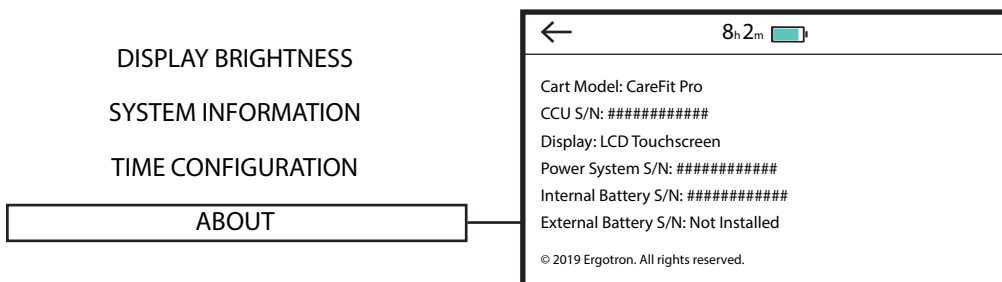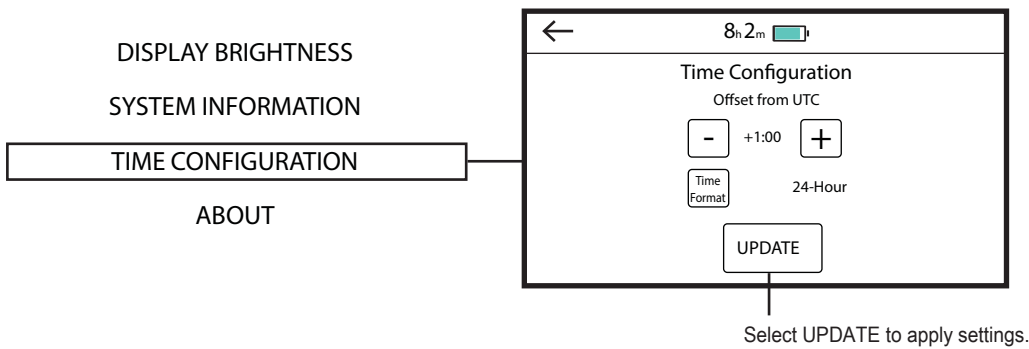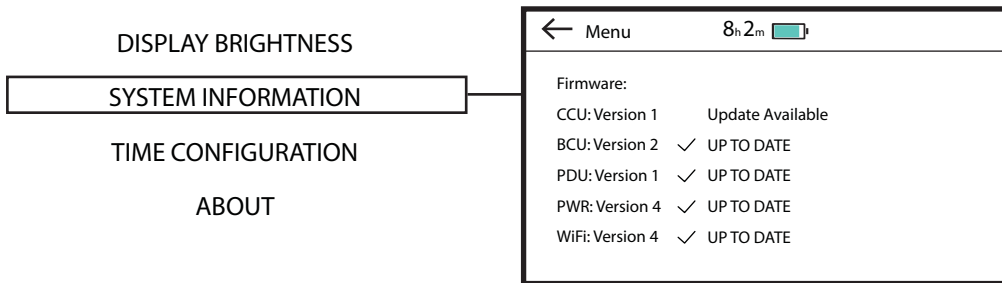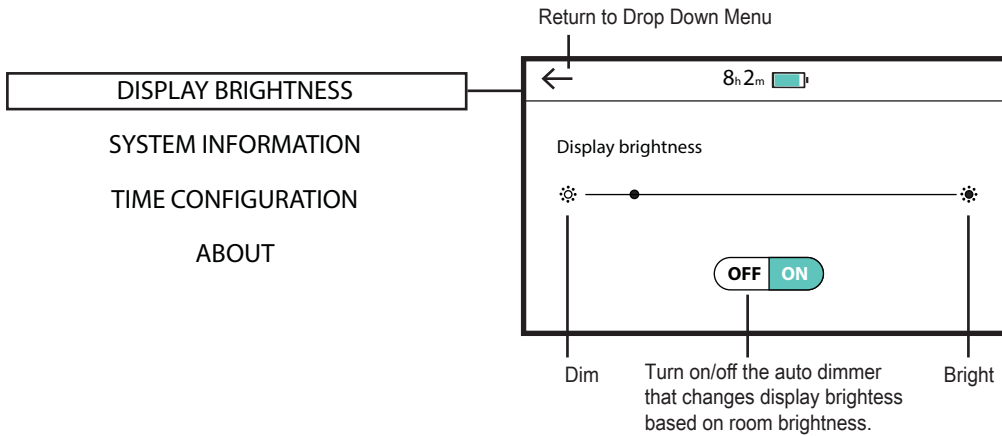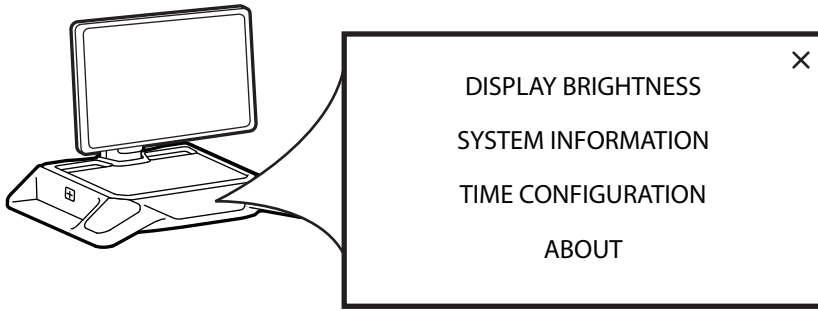



[www.ergotron.com](http://www.ergotron.com) | USA: 1-800-888-8458 | Europe: +31 (0)33-45 45 600 | China: 400-120-3051 | Japan: [japansupport@ergotron.com](mailto:japansupport@ergotron.com)

For the latest User Installation Guide please visit: [www.ergotron.com](http://www.ergotron.com)

For Warranty visit: [www.ergotron.com/warranty](http://www.ergotron.com/warranty)

For Service visit: [www.ergotron.com](http://www.ergotron.com)

For local customer care phone numbers visit: <http://contact.ergotron.com>

© 2019 Ergotron, Inc. All rights reserved. CareFit™ is a registered trademark of Ergotron, Inc.

While Ergotron, Inc. makes every effort to provide accurate and complete information on the installation and use of its products, it will not be held liable for any editorial errors or omissions (including those made in the process of translation from English to another language), or for incidental, special or consequential damages of any nature resulting from furnishing this instruction and performance of equipment in connection with this instruction. Ergotron, Inc. reserves the right to make changes in the product design and/or product documentation without notification to its users. For the most current product information, or to know if this document is available in languages other than those herein, please contact Ergotron. No part of this publication may be reproduced, stored in a retrieval system, or transmitted in any form or by any means, electronic, mechanical, photocopying, recording or otherwise without the prior written consent of Ergotron, Inc., 1181 Trapp Road, Eagan, Minnesota, 55121, USA Patents Pending and Patented U.S. & Foreign. Ergotron is a registered trademark of Ergotron, Inc.