

Zido® Hooks



Product	Part#
Zido Hooks	ZHKzz

NOTE: The Zido cart is not included.

For the latest User Installation Guide please visit: www.ergotron.com



Product	Quantity	Part Number
01 Hooks	2	175-5219-00
02 3/4" Self-threading Screw	2	325-5010-00

STEP 1

Align the Hooks

- Select an installation point for the Hooks.
- Align the hooks with the holes on the Zido tube, top hook facing up and bottom hook facing down.

STEP 2


Install the Hooks

- Attach each Hook to the cart with one self-threading fastener.

Note: These fasteners are creating the threads in the tube, so the first few turns will require some force.



Self-threading Screw
325-5010-00

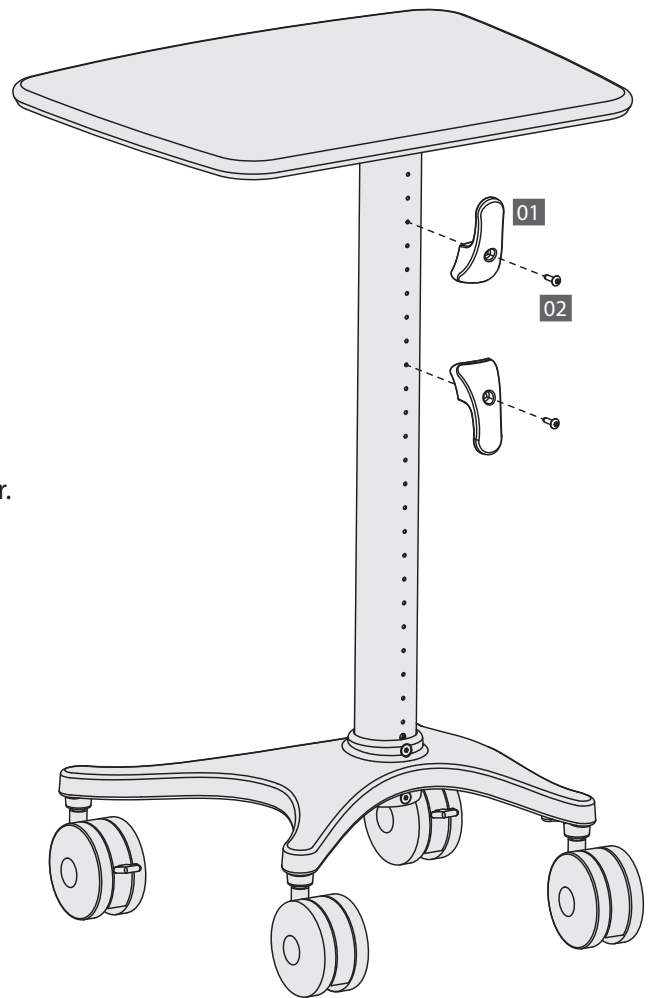

CONGRATULATIONS! YOUR ZIDO HOOK ASSEMBLY IS COMPLETE!

For Warranty visit: www.ergotron.com/warranty

For Service visit: www.ergotron.com

For local customer care phone numbers visit: <http://contact.ergotron.com>

Americas Sales and Corporate Headquarters

1181 Trapp Rd.
St. Paul, MN USA
1-800-325-3841
www.ergotron.com
sales@ergotron.com

ERGOTRON®



While Ergotron, Inc. makes every effort to provide accurate and complete information on the installation and use of its products, it will not be held liable for any editorial errors or omissions (including those made in the process of translation from English to another language), or for incidental, special or consequential damages of any nature resulting from furnishing this instruction and performance of equipment in connection with this instruction. Ergotron, Inc. reserves the right to make changes in the product design and/or product documentation without notification to its users. For the most current product information, or to know if this document is available in languages other than those herein, please contact Ergotron. No part of this publication may be reproduced, stored in a retrieval system, or transmitted in any form or by any means, electronic, mechanical, photocopying, recording or otherwise without the prior written consent of Ergotron, Inc., 1181 Trapp Road, Eagan, Minnesota, 55121, USA. Patents Pending and Patented U.S. & Foreign. Ergotron is a registered trademark of Ergotron, Inc.

© 2016 Ergotron, Inc. All rights reserved.

All trademarks are the property of their respective companies. Anthro is a registered trademark and brand of Ergotron Inc. Anthro-DNA is a trademark of Ergotron Inc.